


How to Insert?

- With clean hands, roll and compress one ear plug into a cylinder with the smallest diameter possible
- Insert tapered end into the ear canal quickly and hold the plug in place till fully expanded
- For a better fit, reach behind the head with your other hand and pull top of the ear while inserting
- Roll the neck loop around your back and repeat for the other ear
- To remove, pull them out gradually with a slow twisting motion, one at a time.

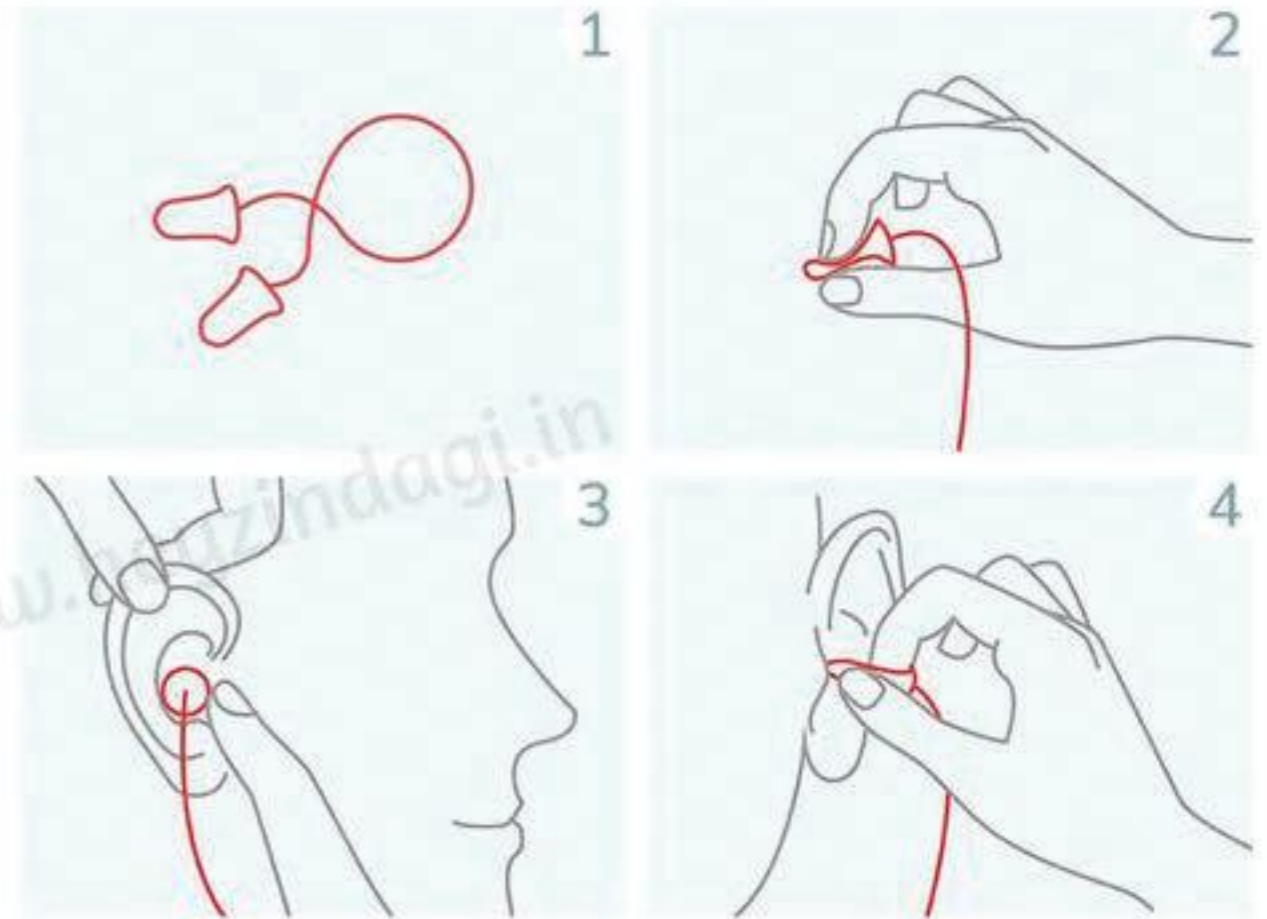

4 Steps to insert and remove the Soft Foam Earplugs.

REMEMBER

- Do not share a used pair - will avoid transfer of infection(s)
- For reuse, store them in a separate box/pouch after wiping with a clean cloth.
- Do not use any that show cracks or are unable to re-expand on pressing

- Not effective for swimming, diving or against loud firearms
- Caregiver must monitor the use of this product by children or adults if they have any physical or mental limitations
- In-Flight: Remove the ear plugs once descent begins

P.S.: Kindly consult your Physician or Specialist before using this product if you have an ear infection, Vertigo or any other condition related to the ear canal