

QUICK START GUIDE

- Make sure the batteries and strap have been correctly installed following the instructions given in user manual (link for download provided in specifications below)
- Go through the parameter setting in the user manual
- Clean the test finger using medical alcohol
- Insert the finger into the Oximeter at the indicated slot as shown in the picture, with the nail surface facing upward
- Ensure that the finger is completely inserted into the rubberised slot of the oximeter
- Record the corresponding measurement data and details shown on the display screen
- Take the finger out of the oximeter
- Clean the rubber hole using medical alcohol before next reading is taken If left unused for 8 seconds, the oximeter will automatically turn off

- Press the Power button on the front case
- Make sure that hand or body is not trembling during measurement



REMEMBER

- Carefully read the user manual in details before operating the oximeter
- Do not use the oximeter in an MRI or CT environment as the magnetic field may interfere with proper functioning of the oximeter
- Try to stay stable during measurement

- Remove fingernail polish or false fingernails before measurement otherwise it may give inaccurate reading
- The pulse oximeter is not suitable for continuous monitoring