

QUICK START GUIDE

- Turn on the thermometer by pressing the ON/SCAN button once
- Press the °C/°F button to select temperature scale of either Celsius or Fahrenheit as per your preference
- Press the MODE button to choose between Body, Room and Surface temperature

Take Body Temperature

- Choose Body Temperature after pressing the MODE button

Take Surface Temperature

- Press the MODE button to switch to Surface temperature
- Press the °C/°F button to select temperature scale of either Celsius or Fahrenheit as per your preference
- Hold the device three to five centimeters from the object or an area
- Press ON/SCAN button once to take the measurement and see the measurement flashing in one second

- Hold the thermometer three to five centimeters away from forehead with the sensor end facing the forehead
- Press ON/SCAN button once to take the measurement and see the measurement flashing in one second
- Note the colour of the backlight to know the level of your body temperature
- Take a 12 to 15 seconds pause between successive measurements

Take Room Temperature

- Press the MODE button to switch to Room Temperature
- Press the °C/°F button to select temperature scale of either Celsius or Fahrenheit as per your preference
- Hold the device away from any surfaces and press ON/SCAN button once to take the measurement

REMEMBER

- The Surface temperature mode can be used to measure the temperature of an object, an area, a food or a liquid. Can also be used to take the temperature of a baby bottle containing milk or water

- The thermometer will turn off automatically after 30 seconds