

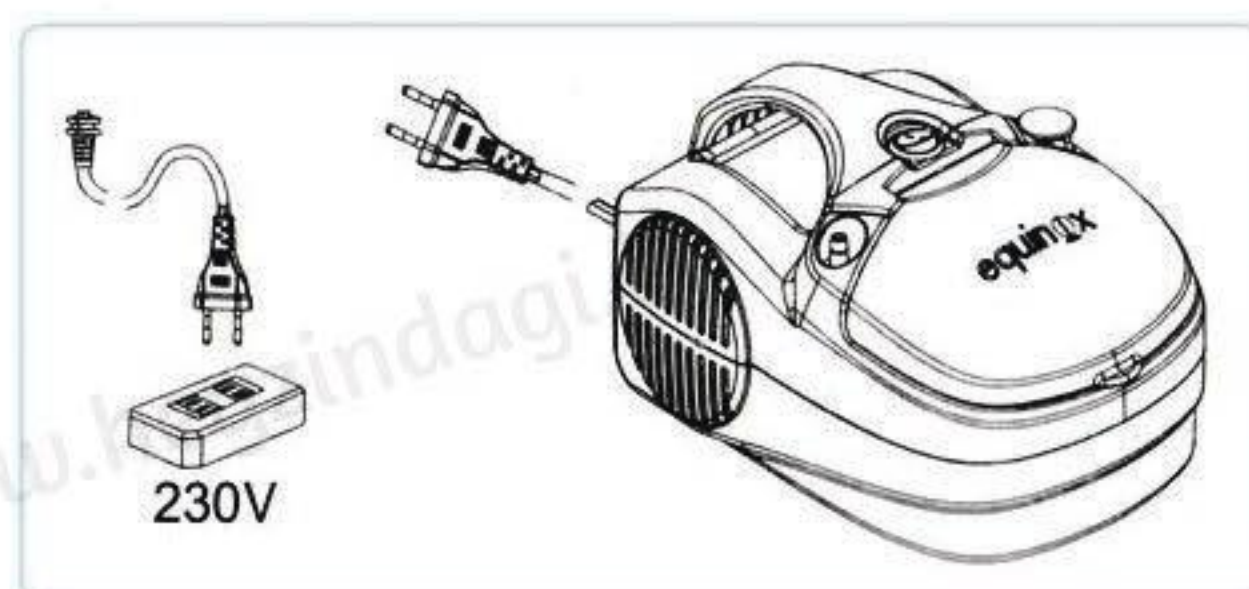


How to Use?

- Place the nebuliser on a flat, stable surface within reach
- Make sure that the nebuliser is clean and the air openings are open and working
- Connect one end of the air tube with the air outlet of the nebuliser and the other end with the medicine chamber [Attention: Do not connect the air tube too tight at either end]
- Put the prescribed medicine in the exact amount as advised by your doctor into the medicine chamber
- Close the chamber correctly
- Insert the power cable into a power outlet to appropriate voltage
- Turn on the main switch
- Be seated comfortably in a stable manner near the nebuliser
- Now turn on the power switch of nebuliser

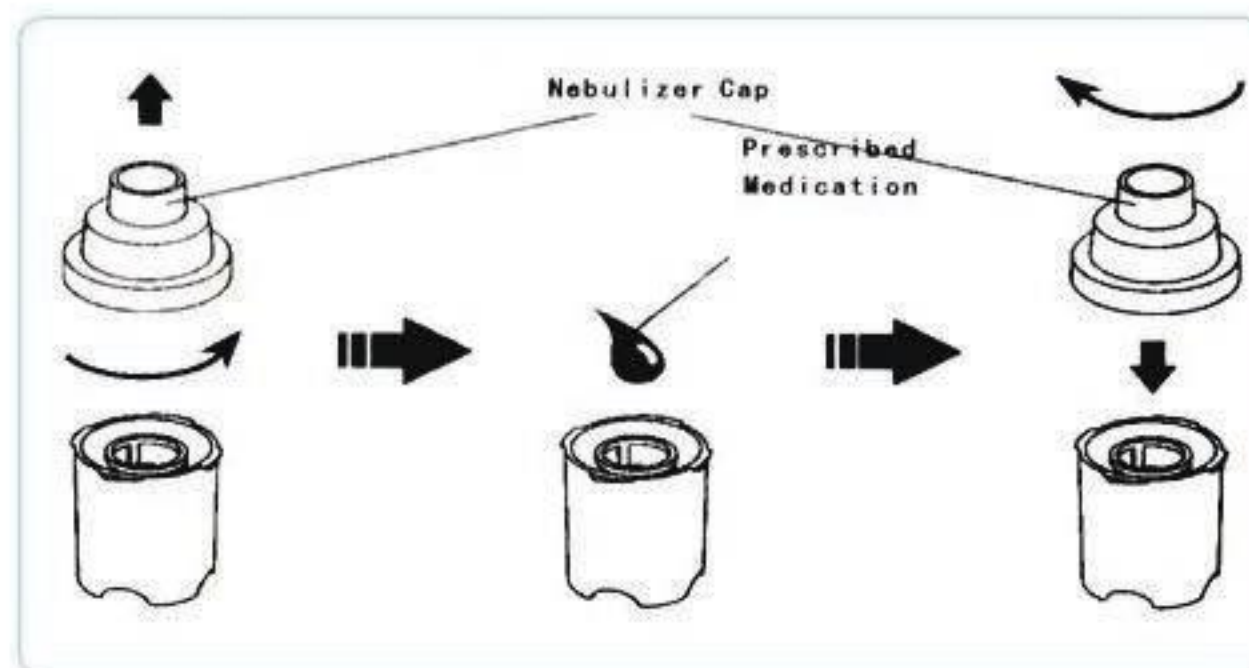

- Notice that the machine instantly turns the medication into the fine mist of aerosolized medicine, which indicates that it is ready to be inhaled
- Position your mouth accordingly with the mouthpiece or the mask as per your convenience and start inhalation
- Turn off the switch after usage

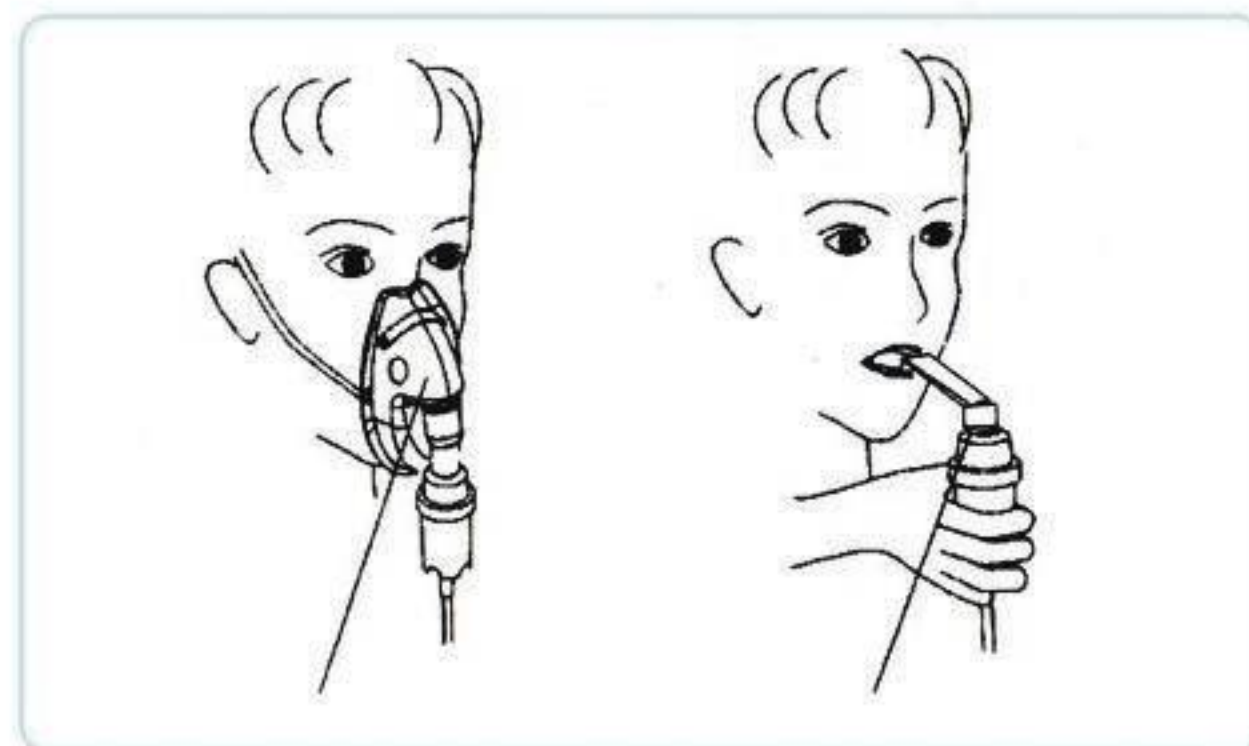

REMEMBER

- **Check the air tube** before each use for any kinks and ensure that there is no leakage
- **Keep the accessories clean and disinfected** for future use. Wash the medicine chamber, mouthpiece and mask with warm water and a mild detergent after every use to avoid cross-infection. Rinse them in hot running tap water. Do not wash in boiling water to avoid damage

- **Replace the filters** at regular intervals to prevent bacterial infections from using an unclean and clogged filter. Open the filter cap and replace the used filter which has turned grey with a new one, for an effective nebuliser treatment
- Keep the nebuliser and other accessories **out of reach of small children** as the small parts might be a choking hazard