

# QUICK START GUIDE

## Step 1:

- Open the lid and inject the drug in the medication chamber/cup
- The filling should not exceed the scale (maximum medication cup capacity is 6ml)
- P.S.- 1. The filling should be done when the chamber is detached from the main unit.  
2. To prevent leakage of medication from the chamber ensure the lid is closed securely

## Step 2:


- Install the medication chamber by sliding it vertically into the main unit and turn on the power by pressing the power button for nebulization (the power indicator will light blue instantly)

## Step 3:

- Hold the nebulizer in your hand stably and start inhalation
- P.S.- Do not cover the vent when you inhale otherwise that will decrease the nebulizer rate

## Step 4:

- Turn the power off if you wish to halt the treatment
- P.S.- 1. The nebulizer can set the working time to 10 minutes on completion of which it shuts off automatically.  
2. The nebulizer shuts off automatically after the medication is depleted

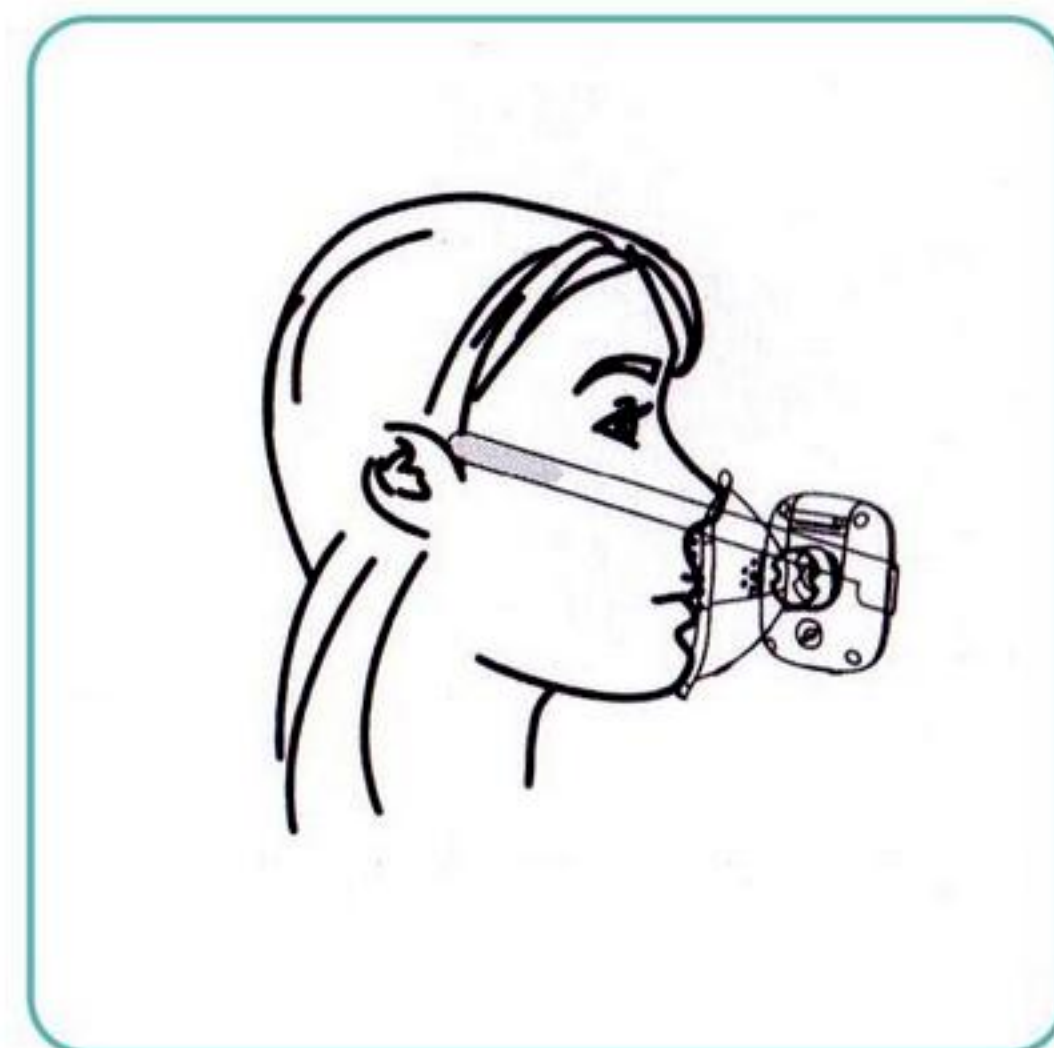

## REMEMBER

- Please recharge the product 30 minutes before first use
- The headband should not be too tight and should be adjusted to suitable tightness

- Disinfect the accessories thoroughly after use
- Read the manual carefully before use