

## How to Use?

- Transporting the user:
  - Pull the chair slightly backwards and push down the wheel attachment to move the chair with the occupant
- In stationary Position:
  - Lock the castors while using in stationary position. Out-turned castors improve the stability of the chair
- Transferring the user:
  - Ensure that all the castors are locked and pointed outwards
  - Remove the armrest on the transfer side of the chair
  - Help the user to be transferred comfortably



### REMEMBER

- The user should not stand on the footrest during transfer as this might cause accidental falls

- Check that the castors have been locked when using the chair over the toilet or keeping it in stationary position. Keep the castors free of hair and soap deposits, etc.