

How to Install?

- Identify the point of installation on the wall as per the user's requirement
 - Height, width and distance of hold play a critical role while installing a grab rail
 - A simple rule of the thumb is - the user must not need to lean over or stretch out while putting weight on the rail, it should ideally be vertically close for an upward/downward motion. If used as a standing aid while bathing working, it should be accessible in a 10 degrees to 30 degrees zone (downward angle) with the arm stretched straight
 - **Ensure that the wall has sufficient load bearing capacity (100 kgs):**
 - Often the tiles/cement simply gives way and the grab bar falls off when the body weight is applied. This is a result of a weak wall and must be protected against prior to installation
 - Kindly get a professional plumber to install this unit and reinforce the material if necessary
 - Inspect the surface to be sure it is completely flat at all fastening points
 - **Cleaning prior to mounting:**
 - Use a sanding sponge with sandpaper surface (80). Clean the mounting surface using the sponge and a detergent (PH 5-9) suitable for the surface. Ensure that the surface is free of limestone residues and dirt
 - Wipe off and dry the surface
 - Wash the surface with the wet wipe supplied to remove any residues from the detergent
 - Wipe the surface with a clean, dry, lint-free cloth
 - Wait until the surface is completely dry and has had time to adapt to room temperature (ideally around 15 minutes)
 - Ensure that the mounting surface is completely clean. If required, other cleaning methods may be used to get rid of residues and dirt
- **Mounting:**
 - Place the glue plate(s) in the handle with the protective tape outwards
 - Remove the protective tape
 - Press the glue plate(s) into place against the wall where the handle is to be positioned
 - Carefully remove the handle(s)
 - Press the glue plates into place to ensure that the tape adheres securely to the wall
 - Insert the tube of glue into the large hole and squeeze glue in until you see glue coming out of the smaller hole (Note: Do not press on the glue plate after it has been filled with glue)
 - Wipe off any excess glue
 - Screw the handle(s) into place using the Allen key supplied. Use the longer screws for the common fixing point(s) of Extensions
 - Indicate date and time when the product is ready for use on the accompanying installation tag and hang it on the product
 - **Removing glue:**
 - Twist the glue plate off the wall using adjustable pliers
 - Scrape off any glue residues with a glass scraper or wash off using silicone remover

REMEMBER

- Ensure that the surface is completely free of limestone residues and dirt. The adhesive effect of the glue cannot be guaranteed if the mounting surface has not been fully cleaned.
- The alcohol cleaning cloth should only be opened it directly before using
- Do not glue the product onto concrete, plaster, wallpaper, veneer or any other covering
- Do not use any other adhesive than provided with the product
- Do not use a screwdriver for securing the handles in position. Use the Allen key provided
- Do not expose the product to any loads or moisture for a period of 24 hours after mounting for allowing the glue time to harden