

QUICK START GUIDE

1. Install 2 AAA batteries into the battery compartment by matching the polarities
2. Clean the test finger using 70% alcohol
3. Hold the oximeter with the display facing you. Press the button to open the probe and slide your finger (with your nails facing upward) into it until the fingertip touches the finger-shaped sensor

P.S:-

For the best result, make sure the finger is centered within the finger guide. Make sure the hand is not trembling during measurement. Ring finger, middle finger and index finger are recommended as suitable monitor positions

4. Press the button to activate the oximeter and then the measurement interface will appear in the display. The measurement result will be read directly from the screen

P.S-To adjust the brightness press the power button for longer than 1 second

The oximeter will turn into sleep mode automatically within 8 seconds after the finger has been removed from the probe



REMEMBER

- Do not use oximeter in a magnetic resonance environment
- The pulse oximeter is intended for use by clinical professionals or under the guidance

- The pulse oximeter is not designed for newborn and infant
- For particular configuration of accessories please refer to the product package list