

How to Use?

- Make sure you order the right fit as per the Size Chart
- Place the device between the Big and Second Toe before putting on your socks or footwear
- Take out the device for a few minutes every day and dry it.
- Also cleanse the area between your toes regularly to avoid any infection or rash
- Check for any signs of skin irritation or allergy if you have sensitive skin



Graphic Courtesy: Oppo Medical Inc.

REMEMBER

- Ensure the toe box of your footwear is comfortable and not too tight
- Avoid direct contact with open wounds

- Replace the unit once the gel hardens over repeated use
- Hand wash the Hallux Straightener, air dry it thoroughly and then apply talcum powder to retain softness

P.S.: If you have sensitive or allergenic skin, kindly consult your Physician or Specialist before using this product