
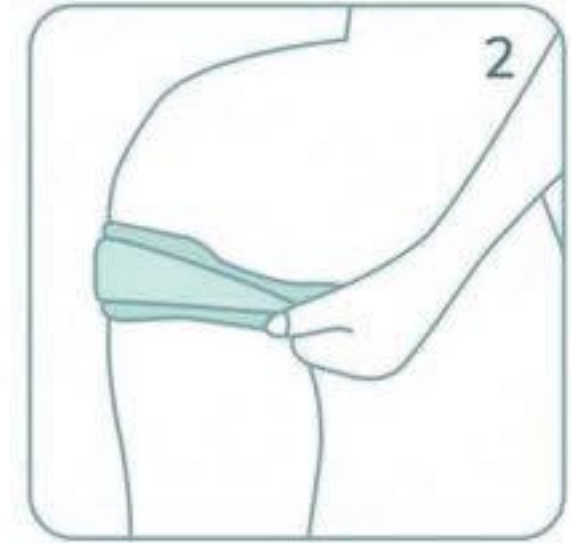


How to Wear?

- Check your Size before ordering the product - it should be between 318 mm to 419 mm as measured around the Knee Joint
- Open both the straps and position the Silicone layer just below the kneecap with the Oppo logo facing upwards
- Secure the main strap with the Hook and Loop fastener
- Wrap the additional strap around the knee and loop it through the metal buckle on top. Pull it add compression as recommended
- Check for comfort, especially circulation. Readjust the fit if necessary

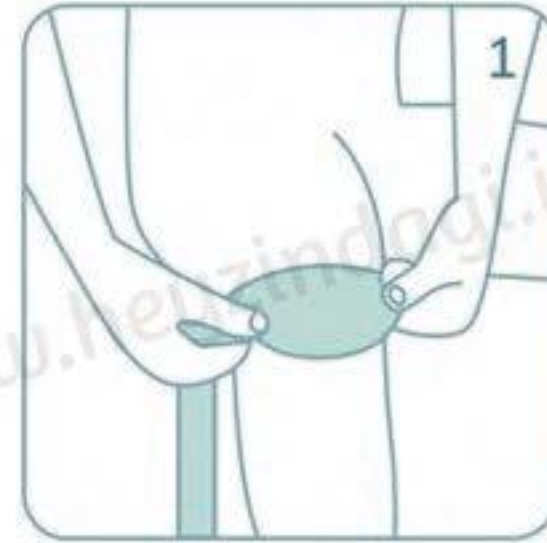

REMEMBER

- Wear this pad while kneeling (if permitted by the Physician) to prevent future injury

- Avoid storing this product with sharp objects to protect the silicone pad

P.S.: Kindly consult your Physician or Orthopaedic Specialist before using this product