


How to Wear?

- Make sure you order the right-sized Belt from the Size Chart below
- Stand in the neutral posture with your back straightened as recommended by the Physician
- Hold the support at your back with the fasteners undone and the Oppo logo facing upward
- Wrap it around your chest and close the fasteners to the compression levels as advised by your Doctor
- Check for all-round comfort
- Please remember to adjust compression as necessary

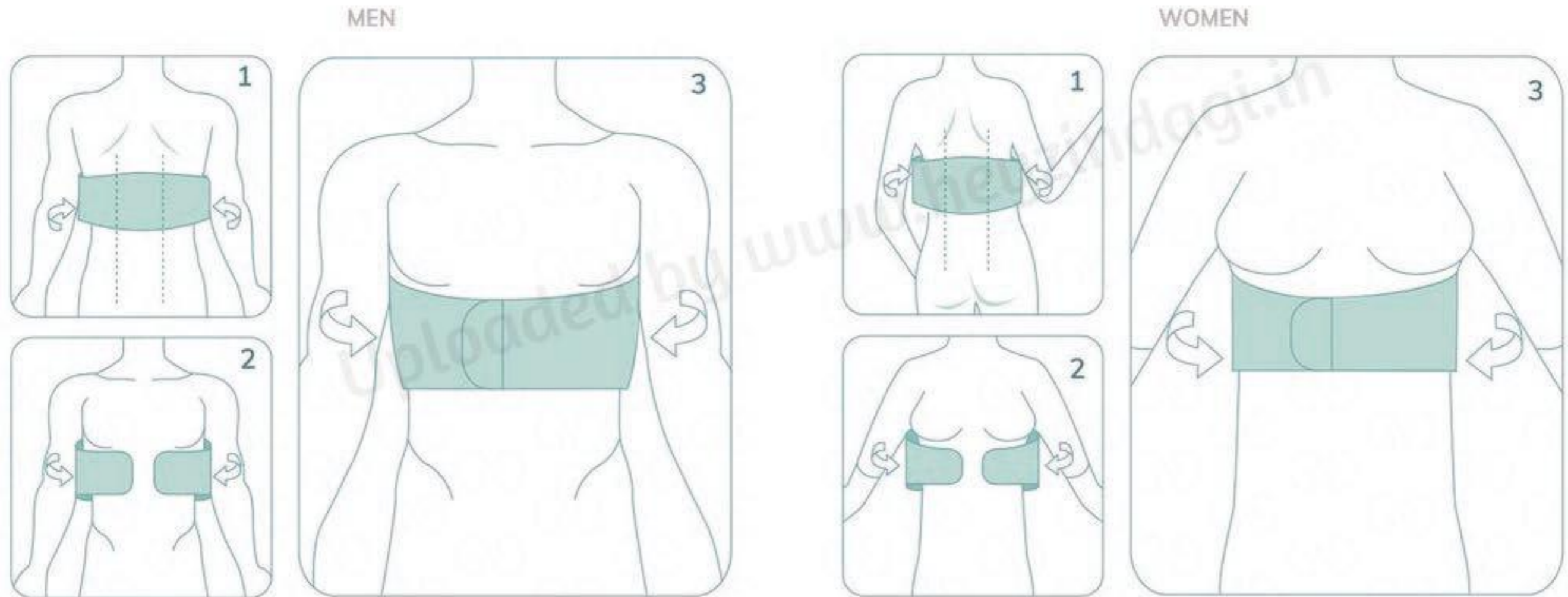

REMEMBER

- Please wear this support above an inner wear made from a breathable fabric such as cotton to avoid rashes and allergy from day long contact
- Do not rely on the support 100% of the time unless advised by a Specialist
- Don't bathe with the support - it will reduce the elasticity and compression quality of the support/belt
- Never share your Orthotic Supports - can lead to a transfer of infection

P.S.: Kindly consult your Physician or Orthopaedic Specialist before using this product