tynor

Functional Knee Support

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Functional Knee Support

English

Functional knee Support, offers the advantage of controlled compression around the knee and a rigid lateral support and immobilization. It allows normal flexion and free movement of the knee joint. Because of front opening is convenient to apply and remove.

Construction:

Made out of strong elasticized fabric. It is shaped anatomically to provide uniform compression around the knee without bunching during knee bending. Strong hinged splints provide lateral stability and support, but allow normal flexion of the knee joint. Anterior opening design allows easy application, particularly in geriatric patients or when the surrounding area is injured or when the knee is swollen or disfigured. Hook loop closures provide controlled compression and close contoured fitting apart from easy application and removal.

APPLICATIONS:

- Support to weak knees
- Control painful knee movements in arthritis
- Patellar dislocation or fracture
- Tendon or ligament injury of the knee
- Muscular sprains and strains of the area
- Post operative care of the knee
- Preventive care in sports activities
- Mild strain and sprain of daily life
- Geriatric care

CAUTION/RECOMMENDATION : Match the axis of the hinge of the splint and knee at the time of application.

INSTRUCTIONS:

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs (risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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

PRODUCT CARE


Hand wash with mild detergent and water below 30°C without wringing.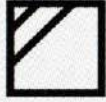

Dry in shade on a flat surface.

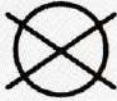Do not dry-clean.

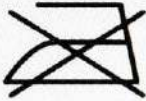Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

CRISIL

W.H.O.
G.M.P.

ISO
13485

CE

US FDA
registered

UM-D09-14-1

www.tynorindia.com