tynor

Ankle Binder

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English

INTENDED USE :

Ankle Binder is an effective device to support, compress and partially immobilize the ankle following injury or sprain to control pain, oedema or inflammation. Composed of two components, compression sleeve and elasticized strap. The sleeve is knitted in 3 dimensions so that it conforms to the complex anatomy of the ankle with exactness.

CONSTRUCTION:

Compression sleeve is knitted on the "state of art", (CNC) three dimensional knitting looms. These advanced looms provide shape conforming to the complex anatomy of the ankle, apart from ensuring localized, variable compression as per the therapeutic requirement.

The elasticized strap is made out of polyester, nylon and rubber yarn of high modulus of elasticity to provide a figure of 8 wrap around the ankle for controlled compression and added immobilization of the ankle. Hook loop closure provide perfect fitting of the binder around the ankle and control of compression, its quick easy application and removal.

APPLICATIONS:

- Sprains and strains of the ankle
- Post operative and post cast care
- Muscle, Tendon and ligament injuries and inflammation
- Oedema
- Contusion
- Arthritis
- Effective as a prophylaxis for sports person

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE


Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.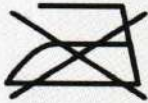

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

Facebook/TynorIndia & YouTube/TynorIndiaOrthotics

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-D01-16-1

www.tynorindia.com