tynor

Anklet/Anklet Comfeel

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Anklet/Anklet Comfeel

English

Anklets are tubular supports widely used in orthopaedic practice to provide mild compression, warmth & support to the ankle joint, to allay pain and inflammation generally associated with old age, arthritis, injury, sports etc.

CONSTRUCTION:

Anklets are made in double layer of Nylon fiber, Spandex and long staple Cotton yarn. Nylon improves life and suppleness of the product. Nylon is aesthetically more pleasing and offers color fastness. Cotton is dermophillic, and hypoallergenic, Interwoven, rubber thread and spandex provide the unique four way stretch to the product.

Anklet comfeel is woven in next generation, computer numeric controlled, 3 dimensional looms, so that the product conforms to the complex anatomy of the ankle with exactness, apart from ensuring localized, variable compression.

APPLICATIONS:

- Gives mild support and immobilization to the weak ankle
- Gives warmth and compression to inflamed ankle joint
- Psychological support to geriatric patients
- For arthritic ankle conditions
- Post-surgical care
- Support during sporting activities

INSTRUCTIONS: ⓘ

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

CRISIL

W.H.O.
G.M.P.

ISO
13485

CE

US FDA
registered

UM-D03/D25-14-1

www.tynorindia.com