tynor

Cast Cover

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Funda De Enyesado

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Note : Please refer to the "How to use" instructions shown on the packaging.

Cast Cover

English

Tynor Cast Cover is a generic cast/bandage/wound protector, which makes the fitting water tight and keeps your cast/bandage/wound dry during the shower or otherwise. It is reusable.

All casts POP or synthetic and the cast padding underneath needs to be kept dry.

CONSTRUCTION:

Tynor Cast cover is made from a durable, transparent PVC fabric. It covers the limb when applied and the sealing is done with an expandable hole drilled into a latex diaphragm which is held in place with a rigid plastic ring.

APPLICATIONS:

- Used as a generic cast/bandage/wound protector, during shower or bathing.
- Can be used as a cast/bandage/wound protector, during transportation in rain/wet weather conditions.

CAUTION/RECOMMENDATION/INSTRUCTIONS :

- For therapeutic care, during shower/bathing only.
- When submersing, do not put weight on plastic ring.
- It is not recommended to be used during recreational water activities.
- With children, adult supervision recommended.
- Be extra careful in wet surroundings, It is slippery when wet.
- Consult your doctor if the diaphragm causes discomfort or swelling.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Wipe with a clean, moist & lint free cloth to clean

Air dry properly, before wearing and after cleaning

**tynor**

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-C16/C19-14-1

www.tynorindia.com