
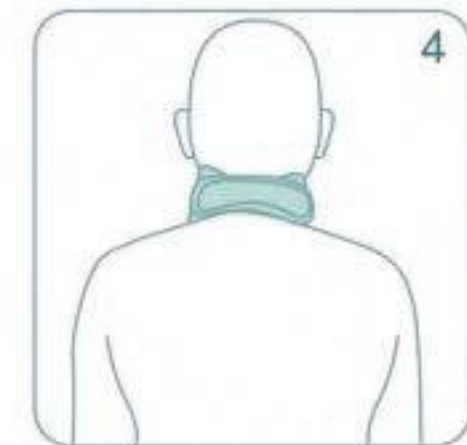


How to Wear?

- Measure your neck size and make sure you order the right-sized Collar from the Size Chart below
- Be seated comfortably and make sure your neck is in a neutral position
- Widen the two parts of the collar to customize it as per the exact measurement of the neck size
- Rest the chin comfortably on the chin cup
- Wrap the collar around the neck snugly
- Fasten the hook tape to the loop tape to a comfortable fit
- Complete the strap closure and readjust if there is any discomfort in breathing, especially pressure on the larynx (Adam's Apple)
Please see the Video in the Product Image Showcase or read Instructions in Download Manual (Specifications Section below) for more

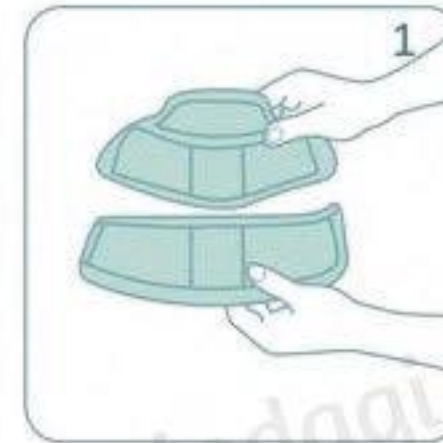

REMEMBER

- The height adjustment in the collar should not be used for sizing purpose. Only to be used for matching the collar height with the height of the neck
- Please avoid wearing any jewelry or dresses with collars that might rub between the collar and your skin often resulting in rashes
- Seek assistance if you are struggling when wearing the collar - it might result in you overstraining your neck

- Discontinue use and consult the doctor in case of impaired sensation, impaired blood flow, allergic reactions on skin and persistence or increase of pain
- Consult the doctor in case of simultaneous use with any other medical device
- Improper use of the device may lead to adverse effects or decreased effectiveness. Follow the instructions on the packaging carefully before using
- Keep it out of reach of children

P.S.: Kindly consult your Physician or Orthopaedic Specialist before using this product