tynor
Ergonomically yours

REF B 20

Cervical Collar Hard with Chin

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Cervical Collar Hard with Chin

English

Cervical Collar Hard with Chin is an excellent device to provide high degree of immobilization and support. It has been designed to improve immobilization by providing the chin lock. The molded chin cup gives an anatomic rest to the head/chin to enhance the comfort. Surprisingly, the collar is extremely comfortable despite made from hard and rigid materials, because of the advanced designing.

CONSTRUCTION:

Cervical Collar Hard with Chin is made out of polyethylene sheet, the edges are padded with molded Ethafoam round tube and covered with vinyl fabric of a superior GSM and quality. Ethafoam has the distinct advantage of a very high cushioning for improved patient comfort, is very light in weight, and has a long functional life. Anatomically molded chin cup is the highlighting feature of this collar, which on one hand provides the chin lock to enhance the immobilization and on the other provides a very uniform resting place for the head/chin to enhance the comfort. Air ventilation holes ensure proper air circulation, thereby improved comfort and compliance. Hook loop closure ensure close fitting of the collar around neck, its quick, easy application and removal. The height of the collar can be varied for accurate dimensional adjustments to suit the individual neck heights.

APPLICATIONS:

Recommended where a rigid immobilization of the cervical region is required

- Cervical Spondylitis
- Cervical disc disease
- Cervical Neuralgia
- Fractures of Cervical Spine
- Post operative care
- Severe trauma to the neck area
- Torticolles
- Post traction stabilization
- Treatment of whip lash injuries
- Other ortho/neuro conditions where rigid immobilization of the neck region is required.

CAUTION/RECOMMENDATION:

Height adjustment in collar should only be used for fine matching of the collar height to the neck height and not for sizing purpose.

INSTRUCTIONS: ⓘ

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Do not wash.

Wipe with a clean moist lint free cloth.

Uploaded by www.heyzindagi.in

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

UM-B20-14-1

ISO
9001

CRISIL

W.H.O.
G.M.P.

ISO
13485

CE

US FDA
registered

www.tynorindia.com