tynor

Cervical Collar Hard Adjustable

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Cervical Collar Hard Adjustable

English

Cervical Collar Hard (Adjustable) used for supporting, immobilizing or adjusting the neck in the flexion, extension, or hyperextension position. Recommended where a rigid immobilization of the cervical region is required.

CONSTRUCTION:

Cervical Collar Hard (Adjustable) is made out of polyethylene sheet, the edges are padded with molded Ethafoam round tube and covered with vinyl fabric of a superior GSM and quality. Ethafoam has the distinct advantage of a very high cushioning coefficient for improved patient comfort, is very light in weight, is hypoallergenic, has good shearing strength apart from being odorless and non absorbent. Air ventilation holes ensure proper air circulation, thereby improved patient comfort and compliance. Hook loop closure ensure close fitting of the collar around neck, its quick, easy application and removal.

The height of the collar can be varied for accurate dimensional adjustments to suit the individual neck heights.

APPLICATIONS:

Recommended where a rigid immobilization of the cervical region is required

- Cervical Spondylitis
- Cervical disc disease
- Cervical neuralgia
- Fractures of cervical spine
- Torticolles
- Early Rheumatoid disease
- Treatment of whip lash injuries
- Other ortho/neuro conditions where rigid immobilization of the neck region is required.

CAUTION/RECOMMENDATION : Height adjustment in collar should only be used for fine matching of the collar height to the neck height and not for sizing purpose.

INSTRUCTIONS: ⓘ

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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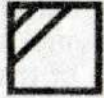
PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-B03-14-1

www.tynorindia.com