tynor

Chest Binder

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Chest Binder

English

Chest Binder is applied to the thoracic region to compress and bind the rib cage and provide splinting to the sternum while allowing sufficient flexibility for comfortable breathing.

CONSTRUCTION:

Made out of superior quality elastic webbing. It contains heat resistant rubber with high modulus of elasticity which ensures a long life of the binder. Broad panels of Hook Loop closure ensure strong and snug fitting of the binder around the chest, its quick easy application and removal.

The chest pad is made out of soft PU and etha foam with high cushioning coefficient to provide effective binding of the rib cage, vibration free splinting and immobilization of the sternum.

APPLICATIONS:

- Stabilization of fractures of sternum
- Post cardio thoracic surgery (Reduce the discomfort of sudden rib cage expansion from deep breaths, cough or sneezing during recovery).
- To protect and bind surgical incisions of chest
- Ease the process of expulsion of phlegm out of lungs after surgery which might otherwise lead to infectious lung pleurisy and other serious complications.

CAUTION/RECOMMENDATION : Center of the Chest pad be located in the furrow of the chest. Compression be carefully controlled to give optimal immobilization and pain relief.

INSTRUCTIONS: ⓘ

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

The logo for Tynor, featuring a stylized wave above the word "tynor" in a bold, lowercase sans-serif font.

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-A11-14-1

www.tynorindia.com