tynor

Contoured L.S. Support

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English

INTENDED USE :

Contoured L.S. Support is scientifically designed to support and immobilize the lumbo sacral region and correct the postural deformity to allay low back pain syndrome. It has rigid, malleable and contoured back splints which can be customized for correct posture and most effective immobilization.

CONSTRUCTION:

Preshaped aluminum splints integrated in the back panel help in providing the anatomically correct support to the lumbo sacral region. It corrects the postural deformity and muscular strains and acts as reinforcement to the back muscles. Made from extra strong orthopedic webbing. Hook loop fasteners facilitates in perfect fitting and adjustment of the belt around the waist, controlled compression, quick easy application and removal. Double pull elastic mechanism allows extra compression and firm grip to the L.S. region.

APPLICATIONS:

- Lumbar spondylitis
- Inter vertebral disc syndrome
- Post discketomy care
- Osteoporetic pain of the lumbar spine
- Low backache, postural fatigue, postural deformities and mechanical stresses due to poor posture
- Comfort during long working hours and travel
- Stabilization during vertebral fracture and injuries
- Degenerative changes of the lumbar spine

▲ CAUTION/RECOMMENDATION :

- Use this product on recommendation of a doctor.
- Contoured LS Support must be individually adapted to the patient by a trained orthopaedic technician, in order to gain maximum immobilization and wearing comfort.

📖 INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs(risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Remove, customize and reinsert the splint for comfortable immobilization.
- Adjust padding and strapping for a secure and comfortable fitting.
- Product be preferably worn over a cotton lining or an under garment.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-4388111
TynorIndia & TynorIndiaOrthotics

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

www.tynorindia.com

UM-A07-16-1