  
**tynor**

*Elastic Knee Support*

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English

## INTENDED USE :

**Elastic knee Support** is a versatile knee support which offers the advantage of knee deloading, lateral immobilization and controlled compression around the knee. It allows normal flexion and free movement of the knee joint.

## CONSTRUCTION:

It is made out of a 250 mm wide, strong elasticized fabric. It is shaped anatomically to compress the knee area and to avoid bunching during bending of the knee. Strong hinged splints provide lateral stability and support, but allow normal flexion of the knee joint. The hinged splints are designed to mimic the controlled motion of the knee. Patellar opening and padding provide proper support and positioning of patella, warmth and comfort to the area. Hook loop closures provide close contoured fitting and a control on the degree of compression apart from easy application and removal.

## APPLICATIONS:

- Support to weak knees.
- Arthritis.
- Patello-femoral problems.
- Tendon or ligament injury of the knee.
- Muscular sprains and strains of the area.
- Post operative care of the knee.
- Preventive care in sports activities or geriatrics .

## ▲ CAUTION/RECOMMENDATION:

- Use this product under strict guidance of a qualified doctor.
- Match the axis of the hinge of the splint and knee at the time of application.

## 📖 INSTRUCTIONS:

- Seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use of another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

**POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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
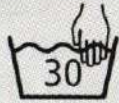
# PRODUCT CARE

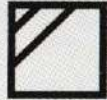Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

**EC REP**

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: [info@mt-procons.com](mailto:info@mt-procons.com)  
[www.mt-procons.com](http://www.mt-procons.com)

## Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

# tynor

## Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: [marketing@tynorindia.com](mailto:marketing@tynorindia.com) • [www.tynorindia.com](http://www.tynorindia.com)

Customer Care : 0091-172-4388111

[Facebook](https://www.facebook.com/TynorIndia) /TynorIndia & [YouTube](https://www.youtube.com/channel/UC...) /TynorIndiaOrthotics

ISO  
9001

W.H.O.  
G.M.P.

ISO  
13485

US FDA  
registered

UM-D08-16-1

[www.tynorindia.com](http://www.tynorindia.com)