tynor

Elbow Support

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Elbow Support

English

Elbow support is widely used in orthopaedic practice to provide firm compression, warmth & support to the forearm and the elbow joint, to allay pain inflammation and stiffness, generally associated with old age, arthritis, sports injury etc.

CONSTRUCTION:

Woven on the state of art CNC looms to provide seam less, tapered tubes, in a double layer of nylon fiber, Spandex and long staple cotton yarn. Nylon improves life and suppleness of the product. Nylon is aesthetically more pleasing and offers color fastness. Cotton is dermophillic, hypoallergenic, warm in winters, cool in summers. Interwoven rubber thread, has a high modulus of elasticity and heat resistance. Intermingled spandex provides the unique four way stretch to the fabric.

The computer controlled knitting offers a unique advantage of soft elbow and fine grip to the edges. It also enhances designing and aesthetics of the product.

APPLICATIONS:

- Helps reduce inflammation and relieves stiffness
- Gives firm compression and therapeutic warmth to inflamed elbow.
- Tennis elbow, Golfers elbow
- Tendonitis
- Offers protection against further injury.
- Psychological support to geriatric patients.
- For arthritic elbow conditions.
- Sports injury, sprains and twists.
- Post-surgical rehabilitation
- Burn care

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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

PRODUCT CARE


Hand wash with mild detergent and water below 30°C without wringing.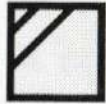

Dry in shade on a flat surface.

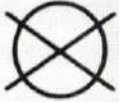
Do not dry-clean.

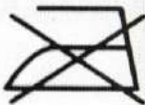Do not Iron.

Do not bleach.

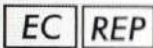MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

tynor

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-5288111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

www.tynorindia.com

UM-E11-17-1