tynor

Foot Drop Splint

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3. Tie the ankle strap tightly around the ankle so as to get proper alignment and grip.

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4. Place the foot and splint into the shoe. Remove the laces completely or wear a slightly bigger shoe to facilitate placement. Tighten the laces and walk freely.

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English

INTENDED USE :

Foot Drop Splint is designed with a dorsal shell made of polypropylene with a leaf spring action to arrest the plantar flexion movement and lift the foot. It supports and stabilizes the ankle and foot in all foot drop conditions. Designed to aid in swing phase toe clearance and to reduce plantar flexion impact in the early stance.

CONSTRUCTION:

Highly anatomic and perforated design. Made out of durable PP polymer. Its thin walled construction has been optimized for increased resilience and designed to be worn with the shoe. Unique leafspring action adds bounce and aids in toe clearance while walking.

Air filled bubble sheet liner ensure soft, comfortable interface with the PP shell. Hook loop bands with reverse buckle mechanism ensure strong grip of the product to the foot.

APPLICATIONS:

- Prevention and correction of foot drop
- Neurological deficit
- Peripheral nerve paralysis
- Peroneal/Peritibial Nerve or Muscle damage
- Ankle or Plantar flexion contracture
- Functional Alignment and stabilization of the foot and ankle
- Post operative care
- Burn patients

▲ CAUTION/RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.
- The contours can be customized using hot air gun to improve effectiveness and comfort.

📖 INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE


Wipe with a clean, moist, lint free fabric to clean.

EC **REP**

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

- LOT** Batch code
 Manufacturing date
 Manufacturer
REF Catalogue number

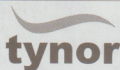tynor

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001-2008

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

www.tynorindia.com