


How to Wear?

- Make sure you order the right-sized product from the Size Chart below
- Maintain the wrist in dorsiflexion (backward bending) position
- Place the hand carefully matching the contours of the hand resting splint
- Adjust the hand and fingers in comfortable position
- Secure the splint to the hand using the hook loop closures

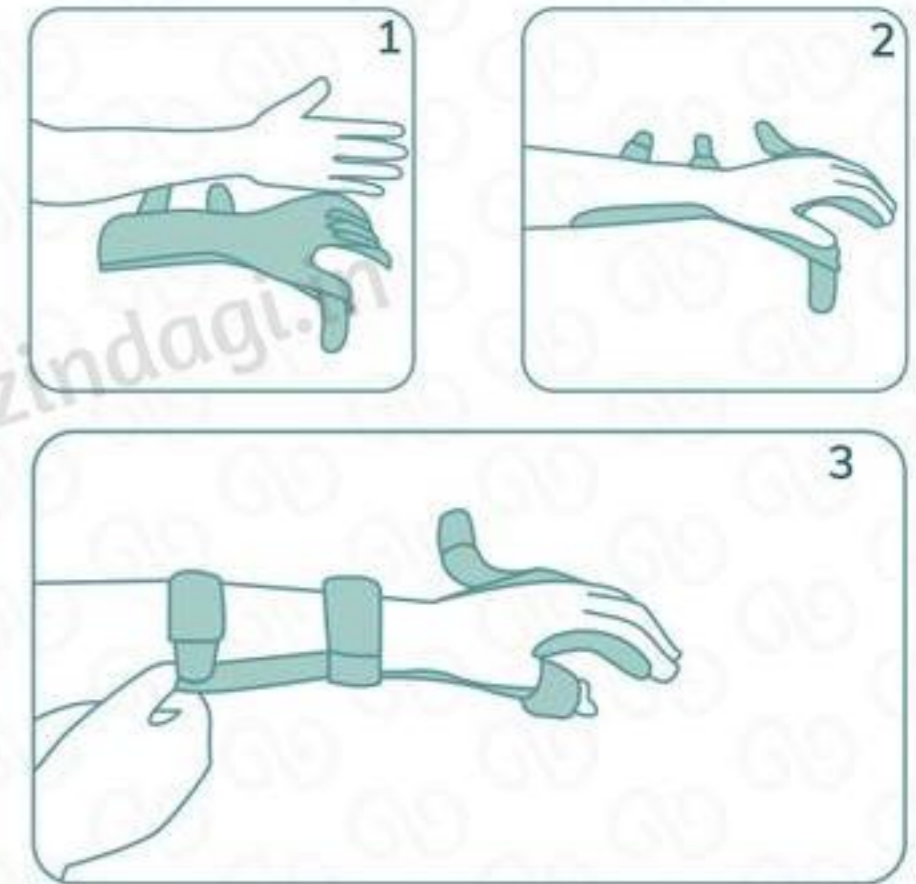

REMEMBER

- Do not roll or bend the splint while using or storing it as it might decrease the effectiveness

- Discontinue use and consult the doctor in case of impaired sensation, impaired blood flow, allergic reactions on skin and persistence or increase of pain

P.S.: Kindly consult your Physician or Orthopaedic Specialist before using this product