tynor

Hand Resting Splint

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Hand Resting Splint

English

Hand Resting Splint is designed to reduce pain and swelling by providing comfortable support and immobilization in the natural position to joints of the hand and the wrist . It reduces irritation of nerves in carpal tunnel by supporting and resting wrist in the "loose pack" position. It maintains length of soft tissues thus prevent contractures.

CONSTRUCTION:

Shell is made from strong molded thermoplastic and lined with Ethafoam for optimum cushioning and comfort. The finger separators help to keep the fingers relaxed and separated to avoid fusion of digits in burn patients. Hook loop strapping provide snug fitting, easy application and removal of the splint. The splint can be removed to permit, therapy sessions, washing and hygiene.

APPLICATIONS:

- For treatment of burns and counteracting contracture
- Hemiplegia, or other types of wrist drop
- Functional hand position, to revive loss of range of motion
- Immobilization and preventing deformity
- Arthritis or other joint inflammation
- Night time splint to reduce pain at night
- Trauma crush injuries
- Carpal tunnel syndrome
- Tenosynovitis
- Tendon rupture
- Post-fracture or surgical care

INSTRUCTIONS: ⓘ

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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Tynor Orthotics Pvt. Ltd.