tynor

Knee Cap (With Rigid Hinge)

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English

INTENDED USE :

Knee Cap (with Rigid Hinge) is a simplified but effective design which offers the advantage of compression around the knee and the support of a side splint. It allows normal flexion of knee.

CONSTRUCTION:

Woven on the state of art circular looms to provide double layered, seam less, tapered made out of nylon and cotton yarn. Interspersed rubber thread, allows uniform compression even on uneven diameters of the limb.

Strong hinges provide lateral support to the injured tendons/ligaments of the knee, while keeping the flexion movement of the knee intact, they mimic the movement of the natural knee joint.

APPLICATIONS:

- Support and warmth to weak or painful knees
- Early stages of Osteoarthritis
- Control painful knee movement in arthritis
- Tendon or ligament injury of the knee
- Post operative care of the knee
- Prophylaxis in sports activities
- Mild strain and sprain
- Geriatric care

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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
PRODUCT CARE


Hand wash with mild detergent and water below 30°C without wringing.

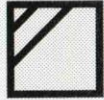
Dry in shade on a flat surface.

Do not dry-clean.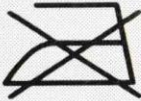

Do not Iron.

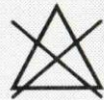Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

tynor

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-5288111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-D06-17-1

www.tynorindia.com