**tynor**

*Knee Support Hinged (Neoprene)*

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## Knee Support Hinged (Neoprene)

English

Knee support Hinged (Neoprene) is a versatile knee support which offers the advantage of controlled compression around the knee and the rigid side support of a splint. It allows normal flexion and free movement of the knee joint.

### CONSTRUCTION:

The body is made out of three layered Neoprene foam bonded fabric, with a four way stretch, the outer layers are made out of strong nylon fabric which ensures a long life, good aesthetics and color fastness. It is ergonomically shaped to provide good grip around the knee anatomy and ensures no bunching on knee bending. The side pockets have aluminum hinged splints to provide lateral stability and support, but allow normal flexion of the knee joint. Anterior Patellar opening and patellar silicon padding provide proper support and positioning of patella and comfortable compression to the area. Hook and loop closures provide close contoured fitting of the brace around the knee and a control on the degree of compression apart from easy application and removal.

### APPLICATIONS:

- Support to weak knees
- Osteoarthritis
- Chondromalacia
- Chronic laxity or instability
- Post cast care
- Post operative care
- Patellar dislocation, fracture or stabilization
- Tendon or ligament injury of the knee
- Muscular sprains and strains of the area
- Prophylaxis during sports activities
- Meniscus Injuries

**CAUTION/RECOMMENDATION** : The product should be removed while sleeping.

### INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

**POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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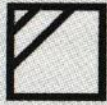
# PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not wring the product

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

**tynor**

**Tynor Orthotics Pvt. Ltd.**

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: [marketing@tynorindia.com](mailto:marketing@tynorindia.com) • [www.tynorindia.com](http://www.tynorindia.com)

Customer Care : 0091-172-4388111

ISO  
9001

CRISIL

W.H.O.  
G.M.P.

ISO  
13485

CE

US FDA  
registered

UM-J01-14-1

[www.tynorindia.com](http://www.tynorindia.com)