tynor

Knee Wrap (Neoprene)

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Knee Wrap (Neoprene)

English

Knee Wrap (Neoprene) is an anterior opening design which offers an advantage of easy application on swollen or asymmetric knees and to geriatrics. It allows customized compression, not possible in tubular products. Neoprene fabric provides firm compression, warmth & support to the knee to allay pain and inflammation, generally associated with old age, arthritis, sports injury etc.

CONSTRUCTION:

Made out of three layered Neoprene foam bonded fabric, with a four way stretch, top layer with unbroken loops offer the advantage of a very versatile closing system and a firm grip. Inner layer made from strong nylon fabric ensures a long life, good aesthetics and color fastness. Neoprene material provides good cushioning and four-way compression which ensure convenient flexion and twisting action to the knee area.

Its anterior opening design and convenient triple hook loop closing system ensure easy application, better grip and controlled compression around the knee.

Patellar opening and patellar padding provide proper support, positioning of patella and comfortable compression to the area.

APPLICATIONS:

- Sprain and strain of the knee
- Support to geriatric knees
- Arthritic knee conditions
- Sports injury
- Post-surgical rehabilitation
- Patellar dislocations
- Patellar tendonitis
- Chondromalacia
- Resistant patello-femoral syndrome

CAUTION/RECOMMENDATION : The product should be removed while sleeping.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not wring the product

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-J05-14-1

www.tynorindia.com