tynor

Lumbo Lacepull Brace

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10.

- Top and the bottom compression can be varied. Pulling on the right lace compresses the top and vice versa, pulling on the left lace compresses the bottom.

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English

INTENDED USE :

Lumbo Lacepull Brace is an advanced brace designed with semi rigid posterior back panel and multiple pulley driven pull mechanism to emphasize controlled and independent pressuring effects, to stabilize lower back pain. It ergonomically braces waist, requires lesser force for old aged users to achieve high compression and immobilization. Recommended specifically for postoperative stabilization and multi-level decompression.

CONSTRUCTION:

Made from aerated mesh fabric which is, light in weight, highly porous and has a strong compression. Anatomical back panel provides perfect fit around the waist for effective decompression, immobilization and enhanced patient comfort.

APPLICATIONS:

- Post-operative stabilization
- Stable, non - displaced spinal fractures
- Spinal stenosis
- Herniated discs
- Degenerative spinal pathologies
- Spondylolithesis
- Spondylolysis
- Ligamentous disorders
- Low back muscle strain

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Product be preferably worn over a cotton lining or an under garment.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Wipe the plastic parts with moist lint free cloth and mild detergent

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

₹1/TynorIndia & /TynorIndiaOrthotics

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

www.tynorindia.com

UM-A29-16-1