tynor

L.S. Lumbopore

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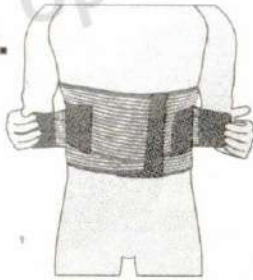
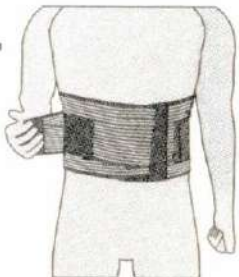
L.S. Lumbopore

English

Lumbopore offers the advantage of advanced designing and the technically correct materials so as to provide the patient effective support and immobilization on one hand and excellent aesthetics and comfort on the other. Lumbopore maintains the lumbar and sacral regions in the neutral position to allay low back pain syndrome.

CONSTRUCTION:

Made from monofilament stripped elastic which is, light in weight, highly porous and has a strong compression. Anatomically streamlined shape provides perfect fitting around the waist, effective immobilization and enhanced patient comfort.

Preshaped semi rigid stays integrated in the back panel help in providing the anatomically correct support to the lumbo sacral region. Spacer fabric provides the requisite cushioning for enhanced comfort.

Double pull elastic mechanism allows extra compression and firm grip to the L.S. region.

Broad hook loop panel facilitates in perfect fitting and adjustment of the belt around the waist, controlled compression, quick easy application and removal.

APPLICATIONS:

- Low backache
- Lumbar spondylitis
- Inter vertebral disc syndrome
- Osteoporetic pain of the lumbar spine
- Degenerative changes of the lumbar spine
- Comfort during long working hours and travel
- Postural fatigue, postural deformities and mechanical stresses due to poor posture

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with another medical device
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Remove, customize and reinsert the splint for comfortable immobilization.
- Adjust padding and strapping for a secure and comfortable fitting.
- Product be preferably worn over a cotton lining or an under garment.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

CRISIL

W.H.O.
G.M.P.

ISO
13485

CE

US FDA
registered

UM-A04-14-1

www.tynorindia.com