**tynor**

**Lumbo Support Uni**

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2.

- The back panel be placed in the centre of the vertebral column, with lower edge of the belt coinciding with the base of the spine.

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3.

- Close hook loop fasteners to a comfortable compression.

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# Lumbo Support Uni

English

**Lumbo Support Uni** is designed for convenience. It supports and immobilizes the lumbo sacral region and correct the postural deformity to allay low back pain syndrome. It has semi rigid contoured splints to enhance comfort on one hand and immobilization on the other.

## Construction:

Made from extra strong orthopedic webbing containing heat resistant rubber with high modulus of elasticity. Large hook loop panel facilitates in perfect fitting and flexibility of sizing. And also controlled compression, easy application and removal.

The tapering shape of the elastic panel at the back helps to cover and immobilize the lumbo sacral region effectively. Preshaped flexible stays integrated in the back panel help in providing the anatomically correct support to the lumbar area on one hand and comfort on the other.

## APPLICATIONS:

- Low back ache
- Inter vertebral disc syndrome
- Lumbar spondylitis
- Osteoporetic pain of the lumbar spine
- Postural fatigue, or postural deformities
- Comfort during long working hours and travel
- Stabilization during vertebral fracture and injuries
- Degenerative changes of the lumbar spine
- Weakened abdominal muscles

**CAUTION/RECOMMENDATION** : This is a general purpose lumbar immobilizer. One size fits all.

## INSTRUCTIONS:

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs (risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Product be preferably worn over a cotton lining or an under garment.

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English

**POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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# PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

**tynor**

**Tynor Orthotics Pvt. Ltd.**

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: [marketing@tynorindia.com](mailto:marketing@tynorindia.com) • [www.tynorindia.com](http://www.tynorindia.com)

Customer Care : 0091-172-4388111

ISO  
9001

CRISIL

W.H.O.  
G.M.P.

ISO  
13485

CE

US FDA  
registered

UM-A15-14-1

[www.tynorindia.com](http://www.tynorindia.com)