tynor

O.A. Knee Support (Neoprene)

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O.A. Knee Support (Neoprene)

English

O.A. Knee Support (Neoprene) is designed scientifically to provide counter force to correct varus or valgus thrust along with some offloading action. It helps to allay pain within a few weeks of use. It can also correct mild to medium varus/ valgus deformity of the knee with a prolonged use.

It offers the advantage of controlled compression around the knee and the rigid side support of a splint. It allows normal flexion and free movement of the knee joint.

CONSTRUCTION:

The body is made out of three layered Neoprene foam bonded fabric, with a four way stretch, the outer layers are made out of strong nylon fabric which ensures a long life, good aesthetics and color fastness.

Anterior opening design allows easy application, particularly in geriatric patients. Hook loop closures provide controlled compression and close contoured fitting apart from easy application and removal.

Strong hinged splints provide lateral stability and support, but allow normal flexion of the knee joint.

APPLICATIONS:

- Mild to moderate osteoarthritis of the knee
- Knee conditions requiring load reduction
- Varus /Valgus thrust
- General pain relief to the knee
- Chronic laxity or instability
- Support to weak or geriatric knee
- Sports injury
- Chondromalacia
- Patella-femoral pain

CAUTION/RECOMMENDATION :

- The product should be removed while sleeping.
- Direction of the counterforce should be understood carefully before application.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case pain increases.
- Discontinue use in case of, allergy rash or itching.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C.

Do not wring the product

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC **REP**

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-J08-15-1

www.tynorindia.com