tynor

Patellar support

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English

INTENDED USE :

Patellar Support is designed to lift the patella and alleviate symptoms of pain, inflammation and discomfort associated with knee degeneration, without restricting circulation.

CONSTRUCTION:

The patellar strap is lined with a plush fabric on the inside to maximize skin comfort. The exterior is lined with a jersey fabric for optimum compression and durability.

The anatomic silicone pad ensures the optimal pressure to the patellar tendon, good grip of the product to the body and absorption of vibration. The hook and loop fastening allows for customized pressure. The pad is dimpled to provide enhanced proprioception to reinforce muscular support to the knee joint.

APPLICATIONS:

- Compresses patellar tendon stabilizes patella and offload patella tendon
- Absorption of vibrations (prevention of the effects of patellar tendinitis, jumper knee)
- Femoropatellar syndrome
- Osgood-Schlatter disease
- Patellar Chondromalacia
- Arthritis, wear and tear of aging
- Enhanced Proprioception reinforces muscular support to knee joint
- Patella subluxation and patella dislocation

CAUTION/RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.
- Make sure your skin is clean and dry before applying product.
- Avoid over tightening, it may restrict blood flow, compress nerve endings, causing swelling, numbness and redness.

INSTRUCTIONS:

- Loosen in case of impaired sensation or impaired blood flow.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

Patellar support

English

INTENDED USE :

Patellar Support is designed to lift the patella and alleviate symptoms of pain, inflammation and discomfort associated with knee degeneration, without restricting circulation.

CONSTRUCTION:

The patellar strap is lined with a plush fabric on the inside to maximize skin comfort. The exterior is lined with a jersey fabric for optimum compression and durability.

The anatomic silicone pad ensures the optimal pressure to the patellar tendon, good grip of the product to the body and absorption of vibration. The hook and loop fastening allows for customized pressure. The pad is dimpled to provide enhanced proprioception to reinforce muscular support to the knee joint.

APPLICATIONS:

- Compresses patellar tendon stabilizes patella and offload patella tendon
- Absorption of vibrations (prevention of the effects of patellar tendinitis, jumper knee)
- Femoropatellar syndrome
- Osgood-Schlatter disease
- Patellar Chondromalacia
- Arthritis, wear and tear of aging
- Enhanced Proprioception reinforces muscular support to knee joint
- Patella subluxation and patella dislocation

CAUTION/RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.
- Make sure your skin is clean and dry before applying product.
- Avoid over tightening, it may restrict blood flow, compress nerve endings, causing swelling, numbness and redness.

INSTRUCTIONS:

- Loosen in case of impaired sensation or impaired blood flow.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-D19-15-1

www.tynorindia.com