tynor

Rib Belt

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Cinturón De Costillas

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English

INTENDED USE :

Rib Belt is applied to the thoracic and upper abdominal region to compress and bind the rib cage during rib fractures and postoperative care, while allowing sufficient flexibility for comfortable breathing.

CONSTRUCTION:

Made out of superior quality elastic webbing with high degree of porosity, it contains heat resistant rubber with high modulus of elasticity, which ensures a long life of the belt without loosening. Hook loop closure ensure proper snug fitting of the belt around thorax, good immobilization, easy application and removal.

APPLICATIONS:

- Rib fracture and dislocations
- Stabilization of fractures of sternum
- Post cardio thoracic surgery care
- To protect and bind surgical incisions of abdomen and chest
- Reduce the discomfort of sudden rib cage expansion from deep breaths, Cough or sneezing during recovery
- Ease the process of expulsion of phlegm out of lungs after surgeries which might otherwise, lead to infectious lung pleurisy and other serious conditions

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE


Hand wash with mild detergent and water below 30°C without wringing.


Dry in shade on a flat surface.

Do not dry-clean.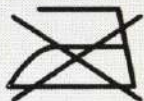

Do not Iron.

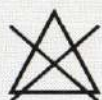Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

www.tynorindia.com

UM-A09-17-1