


## How to Wear?

- Make sure you order the right sized product from the Size Chart given below
- Insert the arm into the sleeve of the support and pull up over the arm and shoulder
- Fasten the hook loop closures of the sleeve to a comfortable fit
- Apply the strap under the opposite arm and close it in the front
- Remember to re-adjust in case of any discomfort

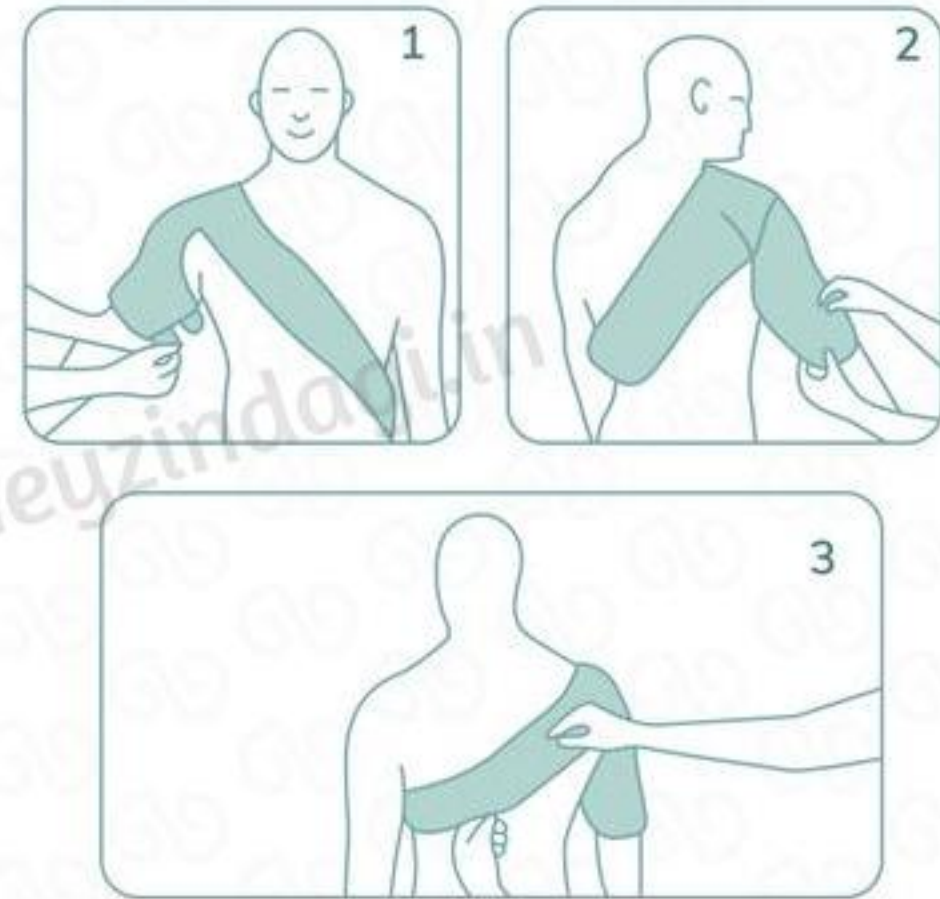

### REMEMBER

- Discontinue use and consult the doctor in case of impaired sensation, impaired blood flow, allergic reactions on skin and persistence or increase of pain
- Consult the doctor in case of simultaneous use with any other medical device

- Misuse or improper use of the product may lead to adverse effects or decreased effectiveness. Please follow the instructions mentioned on the box carefully

**P.S.:** Kindly consult your Physician or Orthopedic Specialist before using this product