tynor

Shoulder Support

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Shoulder Support

English

Shoulder Support is designed to provide protection and support to the injured or ailing shoulder joint, It keeps the shoulder joint warm and flexible, and aids in quick healing.

CONSTRUCTION:

It is designed anatomically to apply uniform and controlled compression around the 'ball' of the shoulder (Deltoid and Rotator cuff region). It is made out of superior quality Neoprene fabric with nylon lining. The design allows for quick adjustment, controlled pressure, easy application and removal.

APPLICATIONS:

- Provides protection, warmth, compression support and flexibility to the shoulder.
- Minor sprains, strains and arthritic pain of the shoulder joint.
- Ideal for the healing and prevention of sporting injuries.
- Arthritis, Bursitis and Tendonitis
- Dislocations and Rotator Cuff Injuries
- Shoulder inflammation, shoulder pain, Mild shoulder instabilities

CAUTION/RECOMMENDATION : Fits either left or right arm

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

tynor

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO
9001

W.H.O.
G.M.P.

ISO
13485

US FDA
registered

UM-J14-14-1

www.tynorindia.com