  
**tynor**

*Taylor's Brace*

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## Taylor's Brace

English

Taylor Brace is a light weight spinal brace, covers all dorsal, lumbar and sacral vertebrae, supports and immobilizes the spine in neutral position, still permitting the requisite body movement.

### CONSTRUCTION:

Made out of malleable, light weight, durable aluminum frame which has two rigid upright para-spinal bars, pre-contoured to keep spine in its natural position. The semicircular splint at the base envelopes hip joint along with the sacrum for rigid immobilization of the entire vertebral column. It can be anatomically shaped for perfect fitting and rigid immobilization.

The frame is lined with Ethafoam padding ensuring excellent comfort. It is then enveloped in a triangular body of 3 layered fabric for enhanced durability and aesthetics. Removable thoracic horizontal splint prevents rotation of upper torso, provides additional immobilization and stability. Twin elastic abdominal panels provide the required compression. Clavicle straps with pads provide good holding and shoulder posture correction.

### APPLICATIONS:

- Thoraco-lumbar injuries
- Inter vertebral disc problems
- Weakened dorsolumbar spine due to tuberculosis
- Post-operative rehabilitation
- Osteoporetic bone collapse in the thoracic and lumbar spine
- Juvenile Scheurmann's disease
- Hyperkyphosis
- Vertebral fracture of the thoracic area
- Any indication requiring dorso-lumbar immobilizations

**CAUTION/RECOMMENDATION :** Both vertical and horizontal splints be removed, customized and reinserted by a professional.

### INSTRUCTIONS

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use of another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

### **POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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# PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing

EC REP

MT Promed Consulting GmbH  
Altenhofstrasse 80  
D-68386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
e-mail: info@mt-procons.com  
www.mt-procons.com

## Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

# tynor

**Tynor Orthotics Pvt. Ltd.**

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-5288111

ISO  
9001

W.H.O.  
G.M.P.

ISO  
13485

US FDA  
registered

[www.tynorindia.com](http://www.tynorindia.com)