tynor

Tummy Trimmer/Abdominal Belt 8"

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English

INTENDED USE :

Tummy Trimmer/Abdominal belt supports and compresses the abdominal muscles for waist line reduction and is used as a post operative or post delivery aid.

CONSTRUCTION:

Made from extra strong highly porous orthopedic webbing containing heat resistant rubber with high modulus of elasticity. Hook loop closures provide perfect fitting of the belt around the waist, controlled compression, its quick, easy application and removal. Flexi Nylon splints ensure no wrinkles, no buckling and no rolling over of the belt.

APPLICATIONS:

- To protect and bind surgical incisions of the abdomen and the chest.
- Postoperative care of thorax and abdomen.
- Reduce the discomfort of sudden rib cage expansion from deep breaths, cough or sneezing during recovery.
- Used as a general tummy trimmer, it compresses the abdominal wall to give the wearer a coveted slim figure.
- For post delivery mothers, to tone up the mellowed abdominal muscles and set the waistline to its Pre-pregnancy dimensions.
- In umbilical and ventral hernia.

RECOMMENDATION :

- Use this product on recommendation of a doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs (risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry one year warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH
 Altenhofstrasse 80
 D-66386 St. Ingbert
 Germany
 tel.: +49 (0) 6894 581020
 fax: +49 (0) 6894 581021
 e-mail: info@mt-procons.com
 www.mt-procons.com

Symbol Explanation :

- LOT** Batch code
- Manufacturing date
- Manufacturer
- REF** Catalogue number

Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India
 e-mail: marketing@tynorindia.com • www.tynorindia.com
 Customer Care : 0091-172-4388111
 Facebook/TynorIndia & YouTube/TynorIndiaOrthotics

ISO 9001 | CRISIL | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

UM-A03-16-1

www.tynorindia.com