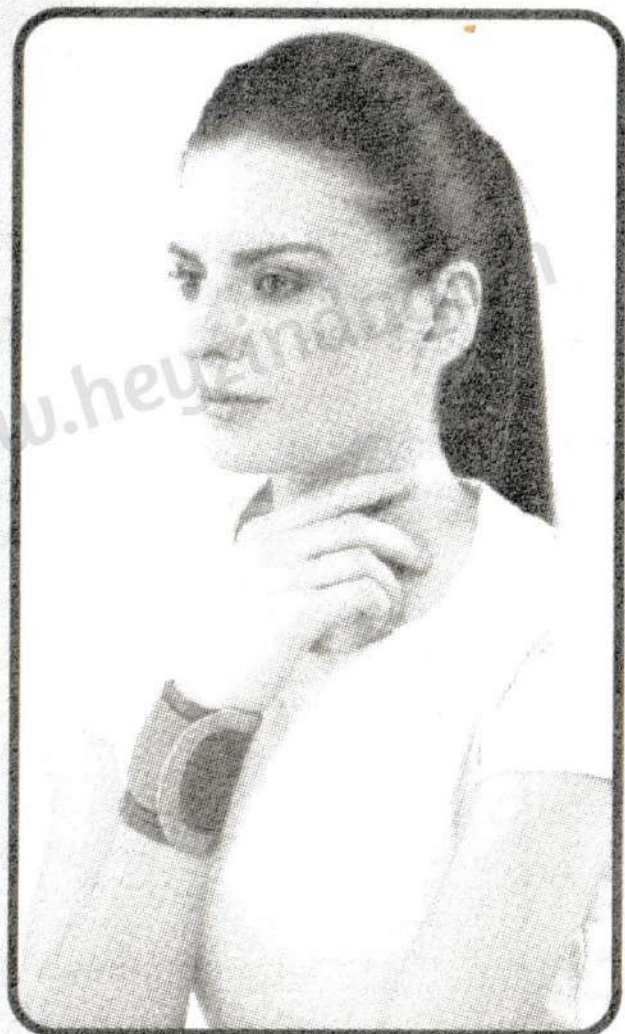
**tynor**

*Wrist Wrap (Neoprene)  
Wrist Brace with Double Lock*

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## **Wrist Wrap (neoprene) & Wrist Brace with Double Lock**

English

### **INTENDED USE :**

Wrist Wrap (Neoprene) and Wrist brace with double lock are designed to support, protect and partially immobilize the wrist. They compress the area and retain the body heat to allay local pain and inflammation.

### **CONSTRUCTION:**

**Wrist Wrap (Neoprene)** is made out of three layered Neoprene foam bonded fabric. The middle layer is made from elasticised neoprene rubber foam, which provides four way stretch to the fabric. The top layer is made out of a very special fabric with unbroken loops. Unbroken loop fabric offers the advantage of a very versatile closing system to provide easy application and a firm grip to the wrist.

**Wrist brace with double lock** is made from strong, porous orthopedic webbing. Double hook loop locking system enhances the immobilization and provides a firm grip to the wrist.

### **APPLICATIONS :**

- Post operative and post cast care.
- Tendonitis, Teno synovitis
- Arthritic care of the wrist
- Prophylaxis in exercise and games
- Carpal tunnel syndrome
- Sprain and Strain
- Dislocation of wrist joint

### **RECOMMENDATION :**

- Use this product under strict guidance of a qualified doctor.

### **INSTRUCTIONS:**

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

**POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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# PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

## Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

**tynor**

## Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-4388111

ISO  
9001

W.H.O.  
G.M.P.

ISO  
13485

US FDA  
registered

UM-J04/E05-15-1

[www.tynorindia.com](http://www.tynorindia.com)