


How to Wear?

- Check your Wrist Size before ordering the product - it should be between 12.5 cm to 23.8 cm
- Insert your thumb into the thumb opening and settle it around the base
- Wrap the product around the wrist
- Fasten the hook to a position of comfortable compression
- Remember to readjust in case of any discomfort

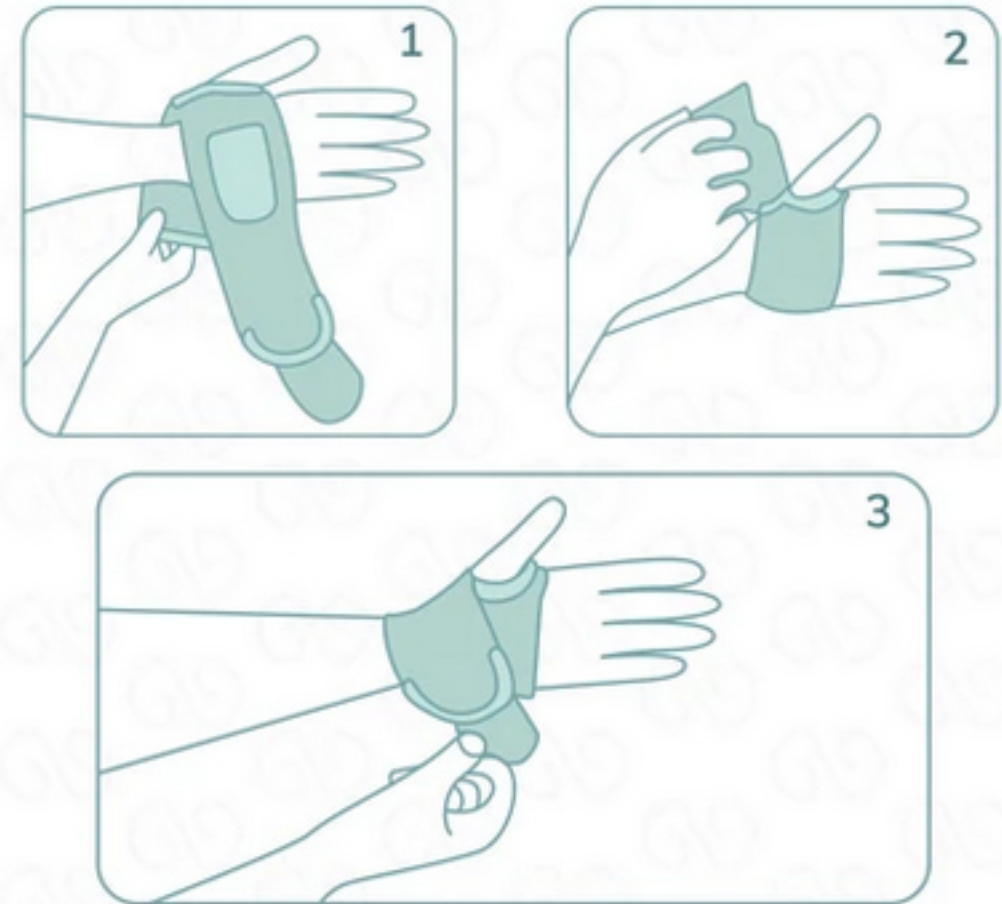

REMEMBER

- Do not roll or bend the support while using or storing it as it might decrease the elasticity and effectiveness
 - Discontinue use and consult the doctor in case of impaired sensation, impaired blood flow, allergic reactions on skin and persistence or increase of pain
- Do not use the product simultaneously with any other medical device. Use or improper use of the product may lead to adverse effects or decreased effectiveness. Please follow the instructions mentioned on the box carefully.

P.S.: Kindly consult your Physician or Specialist before using this product