**tynor**

*Wrist Brace With Thumb*

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English

### INTENDED USE :

Wrist brace with thumb supports, protects and partially immobilizes the wrist while allowing free and natural movement of the thumb and fingers. It compresses the area and retains the body heat to allay local pain and inflammation. It maintains the thumb in its neutral abduction position.

### CONSTRUCTION:

Anatomical thumb opening is designed for slight abduction and comfort of the thumb.

**Wrist Brace with thumb Neoprene :** It is made out of three layered Neoprene bonded fabric, with a four way stretch, top layer with UBL offers the advantage of a very versatile closing system and a firm grip to the wrist. Inner layer made from strong nylon fabric ensures long life and good aesthetics..

**Wrist Brace with thumb (elastic webbing) :** It is made from strong, porous orthopedic webbing, which is hypoallergenic with a good resilience and long life

### APPLICATIONS :

- Post operative and post cast care
- Bursitis, Tendonitis, Tenosynovitis
- Arthritic care of the wrist
- Prophylaxis in exercises and games

### Other Applications for Wrist Brace with thumb Neoprene:

- Carpal tunnel syndrome
- Carpo-metacarpal joint injury
- Sprain and strain
- Ligamentous laxity of thumb

### RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

### INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

### **POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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# PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Do not dry-clean.

Do not Iron.

Do not wring the product : Wrist Brace with thumb Neoprene

EC REP

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

## Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

## Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-4388111

ISO  
9001

W.H.O.  
G.M.P.

ISO  
13485

US FDA  
registered

UM-E06/J03-15-1

[www.tynorindia.com](http://www.tynorindia.com)