  
**tynor**

*Wrist Splint Ambidextrous*

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## Wrist Splint Ambidextrous

English

Wrist Splint Ambidextrous is designed to immobilize and provide firm support to hand and wrist in various orthopedic conditions. It immobilizes the wrist in the dorsi-flexion/ functional position while allowing full range of motion to fingers and thumb. It compresses and retains the body heat to allay the local pain and inflammation. The product can be removed to permit, therapy sessions, washing and hygiene.

### CONSTRUCTION:

It is made from a strong elastic material, which is lined by soft, lycra cotton based extremely dermophillic fabric. The rigid support is provided by anatomically contoured aluminum splints. These splints can be molded in cold to fit the individual patient accurately. Hook loop closures help in easy application and adjustment of the product.

The thumb apertures are designed so that the same product can be used for both left and the right hand, it also maintains the thumb in its abduction position.

### APPLICATIONS:

- Early cast removal
- Fracture management
- Post surgical protection and rehabilitation
- Rheumatic disease of the wrist and arthritic hand positioning
- Bursitis, Tendonitis, Tensosynovitis
- Psychological assurance
- Post extensor tendon repair surgery
- Conservative management of keimbock's disease
- Sprains and strains

### INSTRUCTIONS: ⓘ

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

**POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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

# PRODUCT CARE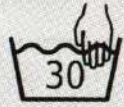


Hand wash with mild detergent and water below 30°C without wringing.

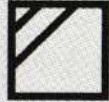Dry in shade on a flat surface.

Do not dry-clean.

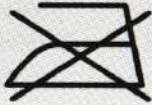Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

## Symbol Explanation :

LOT

Batch code

Manufacturing date

Manufacturer

REF

Catalogue number

**Tynor Orthotics Pvt. Ltd.**

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111

ISO  
9001

W.H.O.  
G.M.P.

ISO  
13485

US FDA  
registered

UM-E43-14-1

[www.tynorindia.com](http://www.tynorindia.com)