






MALEM™ Vibro-Watch 'S' Operation Manual

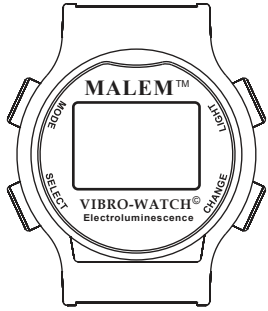

Front View

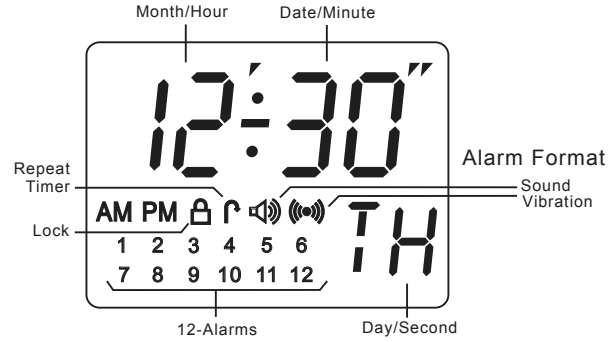


LCD Display

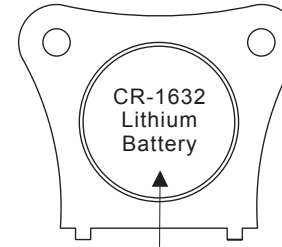


Compartment Label

Battery Door Opener

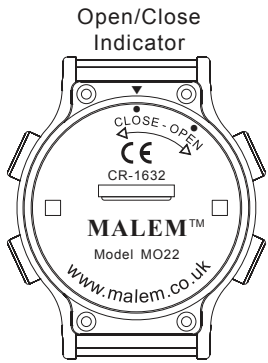

Back View

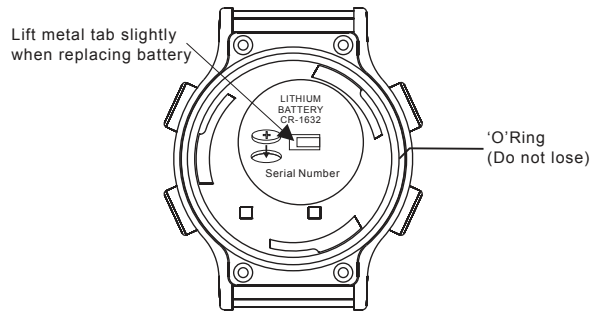


Battery Compartment Label




Battery Door Opener and Battery Holder

To avoid damage use a watch repair shop

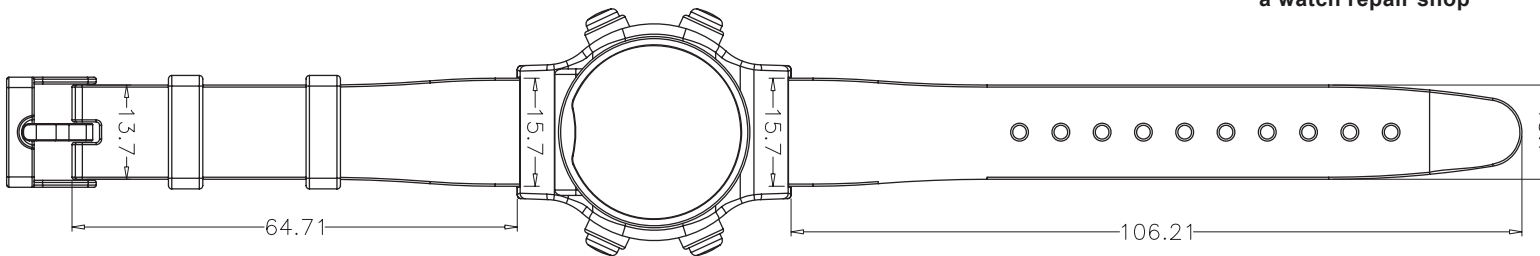

All measurements are show in millimeters

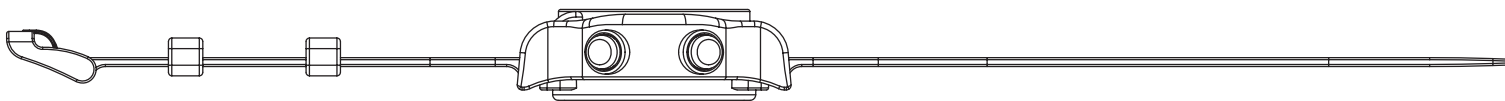

Model MO22



Operation Manual



MALEM Medical Ltd
10 Willow Holt
Lowdham
Nottingham
NG14 7EJ
England, UK
Tel: + 44 (0)115 966 4440
Fax: + 44 (0)115 966 4672
E-mail: malem@malem.co.uk
Web: www.malem.co.uk
Web: www.malemmedical.com


© Malem Medical 2014
Malem Medical Ltd reserves the right to change the specifications without prior notice.

MO22/007/14

IMPORTANT
Please read this instruction sheet before use



Malem™ Vibro-Watch 'S' Operation Manual

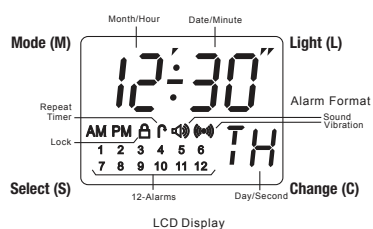


A. Features

- Real Time display in Hour, Minute, Second
- Date Display when (C) key is pressed
- Stopwatch
- Repeat Countdown Timer Alarm with Start/Stop time period setting
- 12 Specific Time Alarms
- Specific Time Alarm and Countdown Timer can be operated simultaneously
- Memory stores all settings when replacing battery. No need to re-programme
- 15 seconds vibration and/or sound alarm format can be selected individually
- Different alarm sounds for Specific Time Alarm and Countdown Timer
- Key Tone Response when operating keys
- Displays Electroluminescence Backlight
- Lock Setting function to avoid accidental change

B. Key Function

- Mode Key (M) - toggle the key to move between modes sequentially

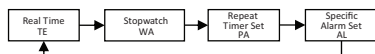

- Select Key (S) - gently press and release key to alter Second and Week-day display mode or select the desired item in setting mode
- Change Key (C) - gently press and release key to show Date or change setting
- Light Key (L) - press once to light up the display for 3 seconds
- Alarm Stop - press any key to stop the active alarm

C. Operating Procedures

1. To set the Date and Time 'TE'

- Gently press (S) Key and hold for 3 seconds until the Second digit flash
- Press (C) Key once to set the digit to zero
- Press (S) Key once Minute digit flash
- Press (C) Key to set correct minute
- Press (S) Key once Hour digit flash
- Toggle (M) Key to select AM/PM or 24 hour format (Military Time)

- Press (C) Key to set correct hour
- Press (S) Key once The Month digit flashes
- Press (C) Key to set correct Month
- Press (S) Key once The Day digit flashes
- Press (C) Key to set the correct Day
- Press (S) Key once The Weekday wording flashes
- Press (C) Key to set the correct Weekday
- Press (S) Key once to go back to the Real Time display TE

2. Stopwatch 'WA'

(i) Normal Counting

- Press (M) Key once to enter Stopwatch mode
- Wording WA displays and then goes into Stopwatch mode
- Press (S) Key once to start the counting
- Press (S) Key again to stop the counting
- Press (C) Key to set reading zero
- Press (M) Key once to go back to the Real Time display TE

(ii) Sectional Counting

- Press (M) Key once to enter Stopwatch mode
- Wording WA display and then goes into Stopwatch mode
- Press (S) Key to start the counting
- Press (C) Key once to read the first sectional time
- Press (C) Key once to recover the time running
- Press (C) Key again to read the second sectional time
- Sectional count can be repeated until the (S) Key is pressed to terminate
- Press (C) Key to set to zero
- Press (M) Key once to go back to the Real Time display TE

3. Periodic Auto-Repeat Alarm (Countdown Timer) 'PA'

(i) To set the Countdown Timer

- Press (M) Key twice to enter the Countdown Timer setting mode PA
- Wording PA displays and then sets page automatically
- Gently press and hold (S) Key for 3 seconds until the minute digit flashes
- Press (C) Key to set countdown period in minute range 0 to 59
- Press (S) Key once and the hour digit will flash
- Press (C) Key to set countdown period hour range 0 to 23
- Press (S) Key once, to set the start Time, 'St' will be displayed (Minutes flash)

- Press (C) Key to set the start countdown time in Minutes (Minutes will flash)
- Press (S) Key once, starting time Hour flash, wording 'St' shown
- Press (C) Key to set the start countdown time in Hours
- Press (S) Key once to set the end time, the wording 'En' will then be displayed (Minutes will flash)
- Press (C) Key to set the countdown end time in Minutes
- Press (S) Key once, to ending time Hour flash, wording 'En' will then be displayed
- Press (C) Key to set the countdown end time in Hours
- Press (S) Key to confirm and starting countdown
- Counting activation is valid within the starting and ending time only
- Maximum countdown time is 23 hours 59 minutes
- Press (M) Key once to go back to the Real Time display TE
- Countdown time can be restarted (Synchronised) by gently pressing the (C) Key for 3 seconds

Note: If the Start Time and End Time are identical, countdown activation function will be continuous.

(ii) To Activate or Terminate the Countdown Alarm PA

- Press (M) Key twice to enter the Countdown Timer setting mode
- Wording PA display and then set page automatically
- Toggle (C) Key to select the alarm format, icon means sound only, icon means vibration only, both icons displayed means both vibration and sound are activated
- Both icons hidden means the Countdown Timer has been terminated, but the setting remains saved in the memory
- Press (M) Key once to go back to the Real Time display TE
- Icon will be shown on display if the alarm had been activated
- When Icon flashes this indicates the timer is counting. Countdown activation is valid within the starting and ending time only
- (iii) View the Setting
- Press (M) Key twice to enter the PA Countdown Timer setting mode
- Wording PA display and then to setting page automatically
- Display showing the countdown status
- Press (S) Key once to show countdown period setting

- Press (S) Key again to show the start countdown time
- Press (S) Key again to show the countdown end time
- Press (M) Key once to go back to the Real Time display
- (iv) Restart (Synchronised) PA Countdown Timer
- Press (M) Key twice
- Wording PA displays and then goes to setting page automatically
- Gently press and hold (C) Key 3 seconds
- The timer resets and counts from the beginning again
- Press (M) Key once to go back to the Real Time display

4. Specific Time Alarm 'AL'

(i) To Set Alarm Time

- Press (M) Key 3 times to enter Specific Time Alarm Setting mode AL
- Wording AL and setting page displays
- Press (S) Key to alter alarm number 1 to 12 until desired number displays
- Gently press and hold (S) Key 3 seconds until minute digit flashes
- Press (C) Key to set the desired minute
- Press (S) Key once alters to the hour digit flash
- Press (C) Key to set the desired hour
- Press (S) Key to confirm the setting
- Press (C) Key to select the alarm format and activate the alarm sound and/or vibration
- Press (S) Key to select another alarm number if necessary adjust as above
- Repeat the above process to set another alarm time
- Press (M) Key once to go back to the Real Time display

(ii) To Activate or Terminate the Specific Time Alarm

- Press (M) Key 3 times to enter Specific Time Alarm Setting mode
- Wording AL and setting page display
- Press (S) Key to alter number 1 to 12 until desired number displays
- Toggle (C) Key to select the alarm format, icon means sound only, icon means vibration only, both icons displayed means both vibration and sound are activated
- If both icons are blank this means this alarm has been terminated but the time setting would not be lost. The settings will be saved in the memory.
- Press (S) Key again to select another alarm number if necessary
- Repeat the above process to activate or terminate another alarm

- Press (M) Key once to go back to the Real Time display
- The activated Alarm Number will be shown on the Time Display

5. Date, Time Display

- Press (C) Key once
- Time display alter to Date and Weekday display
- Display recover to Time display automatically after 5 seconds
- If press and hold the (C) Key, Date and Week-day display will be maintain until C key released

6. Weekday and Second Display Mode Alteration

- Press (S) key once
- The running second digit will alter to weekday display mode
- Press (S) Key again
- The weekday digit will alter back to second display mode

7. Display Back Lighting

- Press (L) Key once
- LCD display lights up for 3 seconds

8. Lock and Unlock Settings

- Press (S) and (M) Key simultaneously to lock all functions
- Icon will be shown on the display
- All setting and functions will be locked
- Press (S) and (M) Key simultaneously once again to unlock all functions
- The icon will then disappear

D. Battery Replacement

TO PREVENT DAMAGE PLEASE USE

A PROFESSIONAL WATCH REPAIR SHOP

Battery: CR-1632 (Battery life about three months when new). Use supplied key to open battery compartment. Please do not damage or lose the rubber 'O' underneath the battery cover. Lift metal contact slightly before inserting fresh new battery. Please observe correct polarity.

E. Guarantee

The Malem™ Vibro-Watch 'S' guarantee covers manufacturing defects for one year from date of purchase. It does NOT cover battery, glass, strap discolouration or damage caused by wear and tear, water, abuse or misuse.

F. Disclaimer

Please note that the Malem™ Vibro-Watch 'S' is designed as a helpful and discreet vibration reminder. It is not intended to replace human care or supervision and the manufacturer, Malem Medical, is not responsible for any consequences resulting from watch use, misuse or failure.