

# Malem Queen's Square Bladder Stimulator

## MODEL PC9-S (9 modes)

To activate  
press pad against skin or  
use switch

Push button down for ON  
Push again for OFF

**Pull Down**  
Use 3 x AA  
Alkaline batteries  
Please observe  
correct polarity  
when replacing  
batteries

Mode  
Button

### How to use

1. Place the Stimulator on the lower abdomen (above the pubic bone) before commencing passing urine and apply gentle pressure to activate the vibration. (If you are unable to exert sufficient pressure to activate, then use the on/off button on the back)

2. Keep the stimulator vibrating while urine is being passed and for about one to two minutes after you feel you have finished. Please select different mode for most effective result. Press & hold mode button for two seconds (while stimulator is on) to activate. Button remains active for sixty seconds.

Adjust rope  
length by tying  
a knot

**Use Alkaline batteries ONLY**