


Adjustable Elastic Mask with Filter Pocket

• beckamade.com • beckamade.etsy.com •



Step 1: Cut rectangle 15" x 8.5"

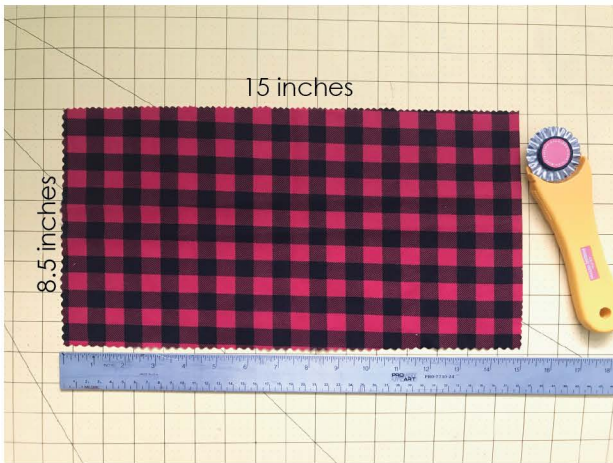

Step 2: Fold right side of fabric together and sew a line on each side, leaving spaces where indicated on the photo.



Step 3: Insert elastic cord cut to 12-13" into the corner openings while still inside out. You can use ribbon or bias tape instead, just cut it longer to allow for it to be tied in a bow.



Step 4: Flip the mask right side out through the 3" opening at the bottom being careful to keep the elastic in place.



Step 5: Tie the elastic in a knot. You can untie and retie it to get the right fit. Then slide the knot into the corner hole if desired to hide it. *optional* cut a filter or blue shop towel to size and insert it into the bottom of the mask.