


# ZVOX Smart UV Dryer

## For Hearing Amplifiers & Hearing Aids

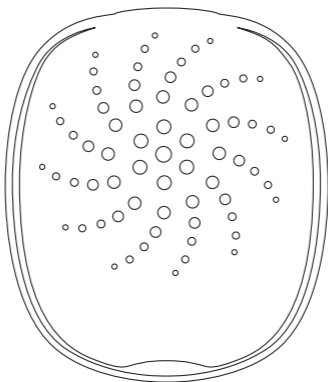

### Operating Instructions

**ZVOX<sup>®</sup>**

Great Sound. Made Simple.

ZVOX.COM

## Operating Instructions

- Connect the USB cable to the power plug.
- Plug mini USB connector into back of dryer.
- Plug the power supply into an AC outlet.

**Note:** only use the supplied power plug.


- Open the lid of the ZVOX Smart UV Dryer.
- Remove the battery from hearing device.
- Place hearing device in Smart UV Dryer.
- Briefly touch the sensor button then close the lid. Red LED light should be on.
- The drying program will start – then stop automatically after four hours.

**Note:** Drying program can be halted by pressing sensor button and holding for more than two seconds. Briefly touch button to restart.

- The red LED light indicates Dryer is on.
- The UV-C light is on for the first five minutes of drying. This shuts down after five minutes.
- Near the end of the four hour drying period, the UV-C light will automatically turn on again for another five minute period.
- After four hours, the UV Dryer automatically shuts down and the LED light will turn off.
- Open the lid and remove the dried and hygienically cleaned hearing devices.
- Use cleaning brush to clean hearing aid/hearing amplifier of any remaining residue – which should now be dry and easy to clean.
- Occasionally use an antiseptic wipe for additional cleaning.

## Maintenance

- We recommend you regularly clean the UV Dryer using antiseptic wet wipes.
- You can stop the UV Dryer by pressing the sensor button for more than two seconds.  
The Smart UV Dryer is quite energy efficient. You do not need to unplug the power supply only for long absences.
- The UV Dryer has an automatic safety function that turns off the UV-C light if the lid is opened during the drying process.
- Low energy consumption: 2.5 W mean power, standby usage less than 0.1 W.

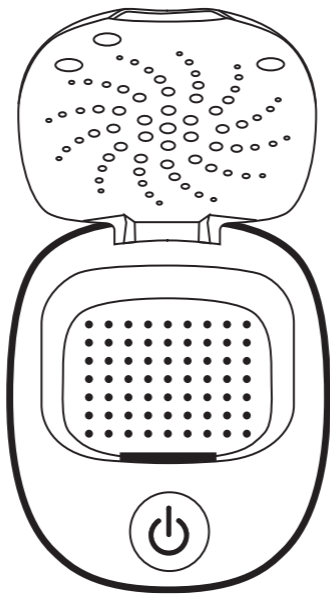

**One Year Limited Parts & Labor Warranty Details:**  
[zvox.com/pages/warranty-repair](https://zvox.com/pages/warranty-repair)

## Warnings

- Don't use the Dryer near water or fire.
  - Do not disassemble the UV Dryer.
  - Only use the supplied power plug/supply.
  - Children must be supervised to ensure that they do not use the UV Dryer as a toy.
  - The UV Dryer should only be operated by those who have sufficient knowledge of appropriate use.
  - CAUTION: Do not look into or touch the UV-C light for an extended period of time.
- 

## Not Ready For Hearing Aids? Try VoiceBuds!



Our dual microphone NoiseBlocker™ system was devised to minimize background sounds – while clarifying the voices you want to hear. Great hearing technology is finally affordable.

- ✓ *Directional microphones reduce background sound.*
- ✓ *AccuVoice® technology boosts voice frequencies.*
- ✓ *Smartphone app for wireless control.*
- ✓ *American made electronic components.*
- ✓ *So small they are virtually invisible when worn.*
- ✓ *Try VoiceBuds For 60 Days Risk Free.*

**60-Day Home Trial | Free Shipping | Free 1 year phone support**

---

**ZVOX®**  
Great Sound. Made Simple.

**ZVOX.com info@zvox.com**