

BION®



PEDAL  
EXERCISER  
FOLDABLE

## INSTRUCTION Manual

### Overview

This product can be used to exercise the legs while sitting on a chair. Place product on the floor at a comfortable distance from the chair. (Not too far away to cause stress or overstretching of the legs.)

Arms can be exercised by positioning yourself comfortably on a chair and product on the table. Securely hold the pedal and you may start pedalling.

### Cleaning

- Wipe with a clean cloth.
- Keep the product dry at all times with a dry cloth to prevent rusting.

### Caution

⚠ Do not stand on the product.  
It is designed to be used in a seating position.

### Warranty Terms & Conditions

([www.bionmedicalgroup.com/warranty](http://www.bionmedicalgroup.com/warranty))

- Product is entitled to 1 years off-site warranty coverage against manufacturing defects from the date of purchase, **with the original invoice/receipt as proof of purchase.**
- This warranty does not cover damages or defects arising from accident, misuse, mishandling, improper installation, any manner of tampering, corrosion/fungus, rusting or stains, any unauthorized repair or modification to the product, act of God, fire, civil unrest and consequential damages.
- This warranty does not cover normal wear and tear.

### Other Information

Product of:

**BION ADVANCE PTE. LTD.**

35 Pioneer Road North #02-03 Singapore 628475


Tel: +65 6250 0562

Email: [info@bionmedicalgroup.com](mailto:info@bionmedicalgroup.com)

Website: [www.bionmedicalgroup.com](http://www.bionmedicalgroup.com)

## Resistance Adjustment

The resistance of the pedals can be increased or decreased by adjusting the knob at the top of the exerciser clockwise or anti-clockwise respectively.

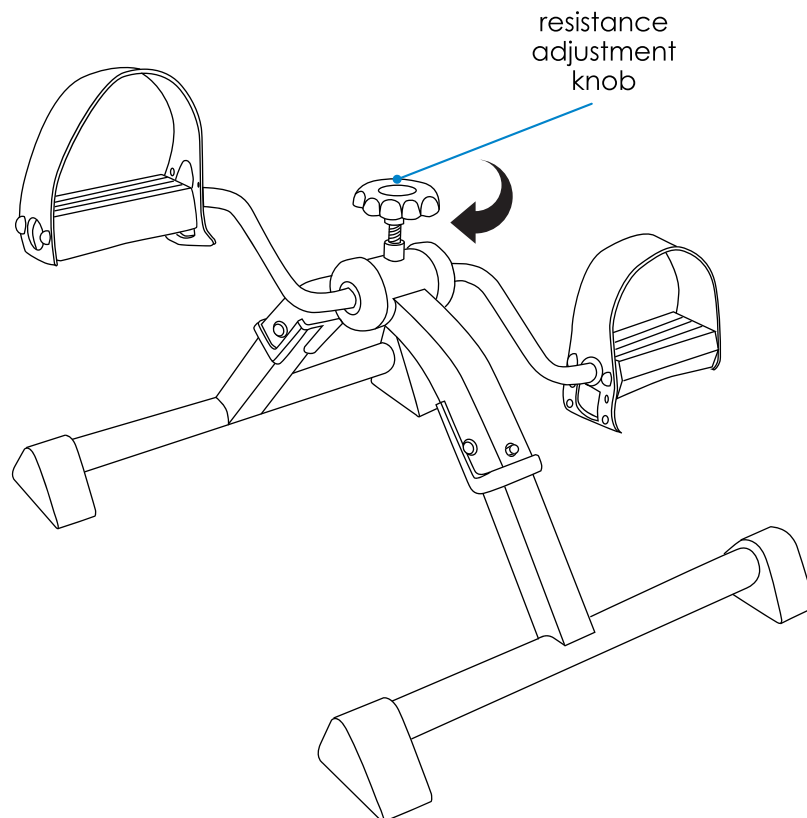

## Compact Storage

This product can be folded for easy transportation and storage. Compact it by depressing the protruding silver button and retract the legs inwards.

