



Bion Safety Bedrail F100 Assembly Instructions

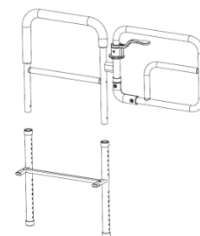


Dear Customers,

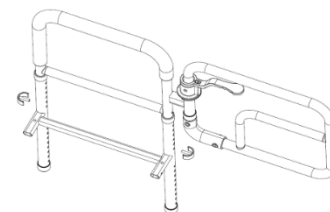
Thank you for using our swing bedrail. Before using the bedrail, kindly read the instruction manual carefully and use it correctly. Keep this manual in a safe place for future reference.

Step 1:

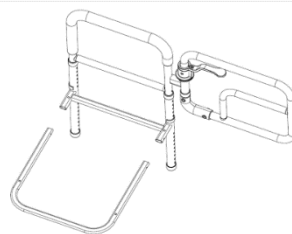
Open the package, take the product out. Align the upper part with the lower support frame. Then insert the upper part into the lower support frame.

**Step 2:**

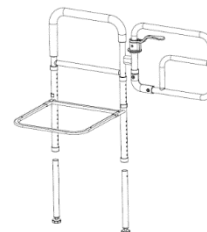
Insert C-shaped clip on both sides.

**Step 3:**

Insert the U-shaped flat pipe, and the marbles on both sides are clamped outside the holes of the support frame.

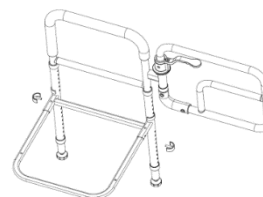
**Step 4:**

Insert the foot tubes on both sides from the bottom.

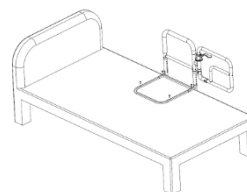


Step 5:

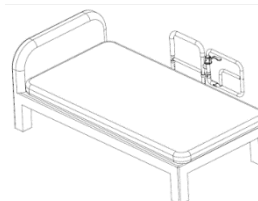
Align the hole position and insert C-shaped fasteners on both sides.

**Step 6:**

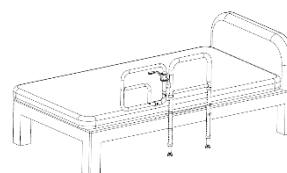
Place the swing bedrail at the appropriate position on the bed, and then fix the swing bedrail on the bed with screws, and fasten it firmly with a tool.

**Step 7:**

Press the mattress on the swing bedrail.

**Step 8:**

Adjust the height of the foot tubes to the ideal height, and then fine tune the foot tubes through the screws to make them flush with the ground.



Step 9:

Pull the switch upward and push it outward to open the door.

