G B R

Read instructions carefully before use. The earplugs must be fitted correctly for them to work effectively. Inserting anything into your ear canal must be done slowly and with care.

Initial Fitting

Earplugs (A) are pre-fitted with the smallest Fit Adjuster (B). Follow the instructions below and try this size before experimenting with the larger Fit Adjusters.

How to Fit

- Step 1) Take the Earplug for the right ear denoted by the letter "R". Check alignment of the Fit Adjuster (B) within the Earplug (A). See fig 1.
- Step 2) Gently push the Applicator (C) to 'slide and clip' onto Fit Adjuster (B) as shown in fig 2. (See fig 6 in the event of difficulty).
- Step 3) **Important:** Double checking the orientation is correct as shown in fig 3. Take the Applicator with the attached Earplug, very slowly insert into the ear canal. (A gentle pull of the earlobe with the opposite hand opens the ear canal making insertion easier).
- Step 4) Remove the Applicator and the Earplug will remain in the ear, perfectly fitted, as shown in fig 4.
- Step 5) Repeat steps 1-4 with the left Earplug denoted by the letter "L".
- Step 6) Once fitted, brush your palm over your ears. If you feel the Earplug's edges, remove the Earplug, check the orientation is correct and re-insert (see figs 3 & 4).

Enlarging The Earplugs To Improve Fitting

If the Earplug feels too loose in the ear canal, the pre-fitted Fit Adjuster may be too small and a larger size is required. Follow the steps below to change the Fit Adjuster to your preferred size.



- Step i) Pull the small Fit Adjusters from both Earplugs to remove. (NB: The silicone material is highly elastic and will stretch without damage).
- Step ii) Having removed the small Fit Adjusters, place the Earplug marked "R" (A), into the Holder (D) as shown in fig 5.
- Step iii) Choose a larger Fit Adjuster (B), insert it into the Applicator (C) as shown in fig 6. (NB: Should the Fit Adjuster fall out or not slide into position, it has been inserted into the Applicator the wrong way up).
- Step iv) With the assistance of the Applicator, insert the larger Fit Adjuster into the Earplug as shown in fig 7 with the triangular head of the Fit Adjuster pointing downwards. Apply enough pressure to push the Fit Adjuster deep into Earplug as shown in fig 8. When satisfied pull the Earplug out of the Holder (D) whilst it remains held by the Applicator. Check the Fit Adjuster is inserted fully and sits in the correct orientation in the Earplug (see fig 1) before proceeding. The earplug is now ready to be fitted.
- Step v) Follow steps 3-6 in the 'How to Fit' instructions above and repeat with the Earplug marked "L" for the left ear. Review the fit and change Fit Adjusters until you achieve a comfortable, secure fit.

Care Instructions

- These Earplugs are made from durable silicon and therefore should be hand washed in mild detergent liquid to remove any residue.
 - To disinfect the ear plugs hand wash in a mild disinfectant and wash thoroughly.
 - This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.
- Recommendations**
- Ensure that the earplugs are fitted, adjusted and maintained in accordance with the manufacturer's instructions.
 - For temporary use only.
 - Ear plugs should be worn at all times in noisy surroundings.
 - The earplugs should be checked on a regular basis for any signs of damage. If damaged the earplugs should not be inserted into the ear canal.
 - If the recommendations are not followed the quality and protection offered by the product may be severely impaired.

Storage

- The Earplugs should be cleaned before and after use and kept in the carry case supplied.

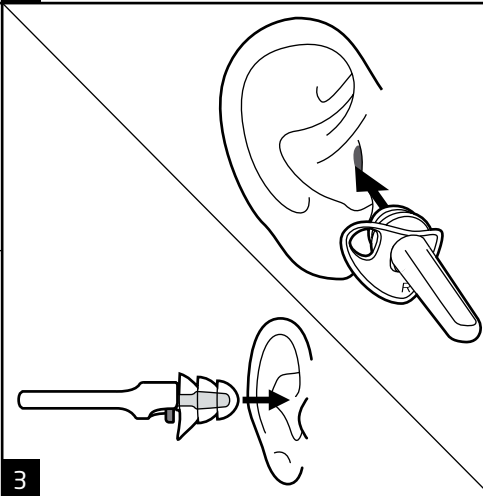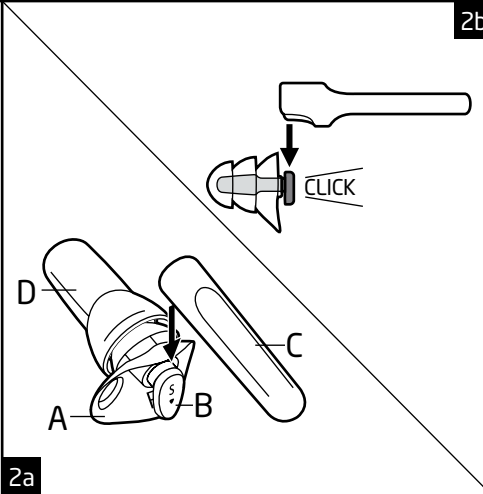A T A

Always insert the Earplug (A) into your ear with the Applicator (B) provided. Do not push the Earplug into your ear canal incorrectly (by hand or otherwise) as shown in Fig 9. The Fit Adjusters should never be inserted into the ears on their own. Keep out of reach of children. Children should be supervised by an adult when fitting and done so in strict accordance with these instructions.

Frequency (Hz)	63	125	250	500	1000	2000	4000	8000
Measured Attenuation (dB)	14.7	16.0	15.7	15.9	18.2	24.0	30.9	34.9
Minimum Permitted Level (dB)	5	9	10	12	12	12	12	12
Mean Attenuation	22.3	22.6	21.6	22.5	24.3	27.7	34.0	40.6
Standard Deviation	7.6	6.6	5.9	6.6	6.1	3.7	3.1	5.7
Assumed Protection	14.7	16.0	15.7	15.9	18.2	24.0	30.9	34.9

M = 25	M = 19	L = 17
SNR = 23		

- Range of nominal diameters - S, M, L 8 - 11. Size XL: 9 - 11.

IMPORTANT WARNING

Always insert the Earplug (A) into your ear with the Applicator (B) provided. Do not push the Earplug into your ear canal incorrectly (by hand or otherwise) as shown in Fig 9. The Fit Adjusters should never be inserted into the ears on their own. Keep out of reach of children. Children should be supervised by an adult when fitting and done so in strict accordance with these instructions. EN352-2 : 2002 Design Go Ltd, Unit 1 Mill Hill Ind. Est. Flower Lane, Mill Hill, London NW7 2HU

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