

venous health measuring charts


MEASURING INSTRUCTIONS

1. Select product/style from measuring chart. 2. Measure at indicated locations on diagram. 3. Locate proper product size based on measurements.

mediven sheer & soft, comfort, for men, active, for women vitality

15–20, 20–30, 30–40 mmHg

Circumference	I	II	III	IV	V	VI	VII
G PANTY & THIGH	16 1/2"–22 1/2" 42–57 cm	17 3/4"–24 1/4" 45–61.5 cm	19 1/4"–26 1/4" 49–66.5 cm	20 1/2"–28 3/4" 52–73 cm	22"–31" 56–79 cm	23 3/4"–33 1/4" 60–84.5 cm	25 1/2"–34 1/2" 64–87.5 cm
C EXTRA-WIDE CALF	N/A	N/A	16 1/2"–19 1/2" 42–49.5 cm	17 3/4"–20 3/4" 45–53 cm	19"–22" 48–56 cm	20 1/4"–23 1/4" 51.5–59 cm	21 1/2"–24 1/2" 54.5–62 cm
C CALF	11"–14" 28–35.5 cm	11 3/4"–15 1/4" 30–39 cm	12 3/4"–16 1/2" 32.5–42 cm	13 3/4"–17 3/4" 35–45 cm	14 1/2"–19" 37–48.5 cm	15 1/2"–20 1/4" 39.5–51.5 cm	16 1/4"–21 1/2" 41.5–54.5 cm
B ANKLE	7"–7 3/4" 18–20 cm	8"–8 3/4" 20.5–22.5 cm	9"–9 3/4" 23–25 cm	10"–10 3/4" 25.5–27.5 cm	11"–11 3/4" 28–30 cm	12"–13" 30.5–33 cm	13 1/4"–14 1/4" 33.5–36 cm
LEG LENGTH	∟ floor - D1 CALF ▶ STANDARD: 15"–19" (38–47.5 cm) PETITE: 12"–15" (31–38 cm) TALL: >18" (>45.7 cm) ∟ floor - G PANTY & THIGH ▶ STANDARD: 28"–34" (71–86.5 cm) PETITE: 23"–28" (58.5–71 cm)						

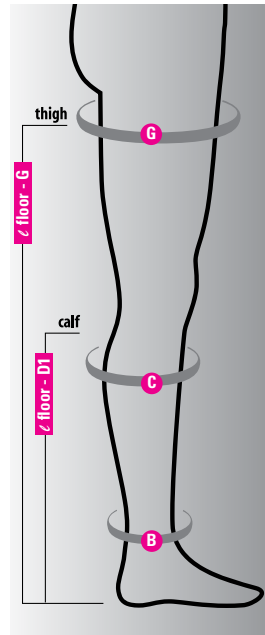

mediven plus, forte 20–30, 30–40, 40–50 mmHg

Circumference	I	II	III	IV	V	VI	VII
G PANTY & THIGH	15"–22 1/2" 38–57 cm	16"–24 1/4" 41–62 cm	17 1/4"–26 1/4" 44–67 cm	18 1/2"–28 3/4" 47–72 cm	19 3/4"–30 1/4" 50–77 cm	21"–32 1/4" 53–82 cm	22 1/4"–34 1/4" 56–87 cm
G THIGH w/Silicone Top Band	16 1/2"–19 1/4" 42–49 cm	17 3/4"–20 3/4" 45–53 cm	19"–22 1/4" 48–57 cm	20 1/2"–24" 52–61 cm	22"–25 3/4" 56–65 cm	23 1/2"–27 1/2" 60–70 cm	25"–29" 64–74 cm
C EXTRA-WIDE CALF w/Silicone Top Band w/out Silicone Top Band	13 3/4"–15" 34–38 cm	14 1/2"–16 1/2" 37–42 cm	15 3/4"–18" 40–46 cm	17"–19 1/2" 43–50 cm	18"–21" 46–54 cm	19 1/4"–22 3/4" 49–58 cm	20 1/2"–24 1/2" 52–62 cm
C CALF & CALF w/Silicone Top Band w/out Silicone Top Band	10 3/4"–13 3/4" 26–34 cm	11"–14 1/2" 28–37 cm	11 3/4"–15 3/4" 30–40 cm	12 1/2"–17" 32–43 cm	13 3/4"–18" 34–46 cm	14 1/4"–19 1/4" 36–49 cm	15"–20 1/2" 38–52 cm
B ANKLE	6 1/2"–7 1/4" 16.5–18.5 cm	7 1/2"–8 1/4" 19–21 cm	8 1/2"–9 1/4" 21.5–23.5 cm	9 1/2"–10 1/4" 24–26 cm	10 1/2"–11 1/4" 26.5–28.5 cm	11 1/2"–12 1/2" 29–32 cm	12 3/4"–13 3/4" 32.5–35 cm
LEG LENGTH	∟ floor - D1 CALF ▶ STANDARD: 15"–19" (38–47.5 cm) PETITE: 12"–15" (31–38 cm) ∟ floor - G PANTY & THIGH ▶ STANDARD: 28"–34" (71–86.5 cm) PETITE: 23"–28" (58.5–71 cm)						

For All Panty & Thigh Styles:

Measure **B C G**
and ∟ floor - G

For All Calf Styles:

Measure **B C**
and ∟ floor - D1


mediven for men classic 8–15 mmHg Calf

Circumference	MEDIUM	LARGE	XLARGE
SHOE SIZE	8–10	10 1/2–12	12 1/2 AND OVER

mediven sheer & soft 8–15 mmHg Calf

Circumference	SMALL	MEDIUM	LARGE	XLARGE
SHOE SIZE	4–5	5 1/2–7 1/2	8–10	10 1/2–12

mediven sheer & soft 8–15 mmHg thigh and panty

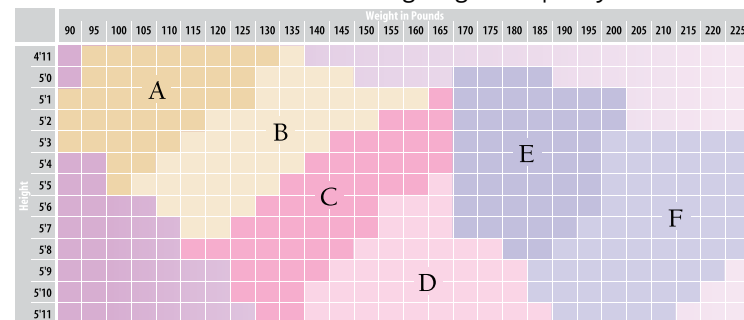

venous health measuring charts continued

assure by medi and mediven dual layer stocking system

15–20, 20–30, 30–40, 40–50 mmHg

Circumference	SMALL	MEDIUM	LARGE	XLARGE	XXLARGE
G PANTY	20"–26" 51–66 cm	22"–28" 56–71 cm	24"–30" 61–76 cm	26"–32" 66–81 cm	N/A
G THIGH w/Silicone Top Band	19 1/2"–24 1/2" 49.5–62 cm	21 1/2"–26 1/2" 54.5–67 cm	23 1/2"–28 1/4" 60–72 cm	25 1/2"–30 1/4" 60.5–77 cm	N/A
C EXTRA-WIDE CALF	N/A	N/A	17"–20" 43–51 cm	18"–22" 46–56 cm	20"–24" 51–61 cm
C CALF	11"–15" 28–38 cm	12 1/4"–16 1/2" 31–42 cm	13 1/2"–18" 34–46 cm	14 1/2"–19 1/2" 37–50 cm	18"–22" 46–56 cm
B ANKLE	7"–8 1/2" 18–21.5 cm	8 1/2"–9 3/4" 21.5–25 cm	9 3/4"–11" 25–28 cm	11"–12 1/2" 28–32 cm	12 1/2"–14 1/2" 32–37 cm
LEG LENGTH	∟ floor - D1 CALF ▶ STANDARD: 15"–19" (38–47.5 cm) PETITE: 12"–15" (31–38 cm) ∟ floor - G PANTY & THIGH ▶ STANDARD: 28"–34" (71–86.5 cm) PETITE: 23"–28" (58.5–71 cm)				

NOTE: The assure by medi product is available in both standard and petite lengths. The mediven dual layer stocking system products are only available in standard lengths.

duomed size charts

MEASURING INSTRUCTIONS

1. Select product/style from measuring chart. 2. Measure at indicated locations on diagram. 3. Locate proper product size based on measurements.

duomed transparent, duomed advantage, duomed freedom, duomed patriot, and duomed relax

Circumference	S	M	L	XL	XXL
G PANTY	20–26" (51–66 cm)	22–28" (56–71 cm)	24–30" (61–76 cm)	26–31.5" (66–80 cm)	27.5–32.35" (70–82 cm)
G THIGH w/ TOP BAND	19.25–24.5" (49–62 cm)	21.25–26.75" (54–68 cm)	23.75–29" (60–74 cm)	25.5–30.25" (65–77 cm)	27.5–32.35" (70–82 cm)
C CALF	11–15" (28–38 cm)	12–16.5" (31–42 cm)	13.5–18" (34–46 cm)	14.5–19.75" (37–50 cm)	17.5–21.5" (44–54 cm)
B ANKLE	7.5–8.75" (19–22 cm)	8.5–9.75" (22–25 cm)	9.75–11" (25–28 cm)	11–12.25" (28–31 cm)	12–13.5" (31–34 cm)
Leg length	STANDARD		PETITE		
CALF	∟ floor - D1		15–17.5" (39–44 cm)	13–15" (34–38 cm)	
PANTY & THIGH	∟ floor - G		28.25–32.75" (72–83 cm)	24.25–28" (62–71 cm)	

For All Panty & Thigh Styles:

Measure **B C G**
and ∟ floor - G

For All Calf Styles:

Measure **B C**
and ∟ floor - D1

